Q1: Missing parts when received.

A: If you find some parts are missing from the box, please kindly send us messages to let us know the name of missing part regarding to the first page of instructions. We will send replacement to you asap.

Q2: Some parts damaged when arrived.


A: Please send clear photos about damaged part to us as attachment, we will check and offer you best solution soon.

Q3: The chair is stuck on lifting or reclining position, and no response when you press the lifting or reclining button.


A: Please kindly check if the power cord (E) are connected well with lift motor. Sometimes the power cord (E) may fall off from the motor.

If the connection is good, then check the lift motor if it has some anomaly, like some noise comes out when you press the lift button. If you can make sure the motor is defective, please contact us for replacement.

How to remove the chair? Please note that you need to take apart the backrest before.

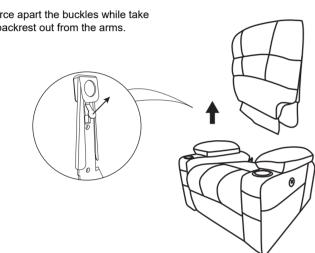

Please make sure you've got everyting lists as following before assembly.

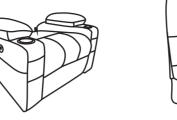

A Chair base



B Backrest


Power Source








Power Lift Chair Assembly Instruction

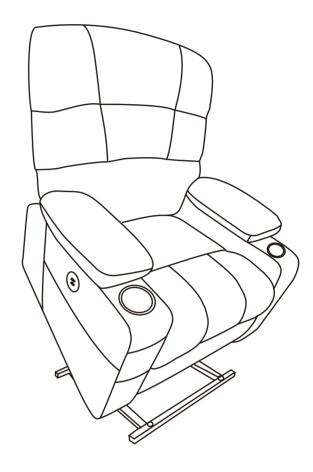

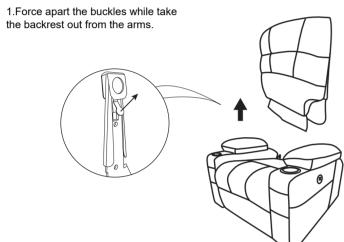


■ Remote control


G Supporting Feet x4

STEP 1

Find a small bag in the accessories box. Take out Supporting Feets from the small bag. Assemble the four Supporting Feet as shown on the bottom of mechanism

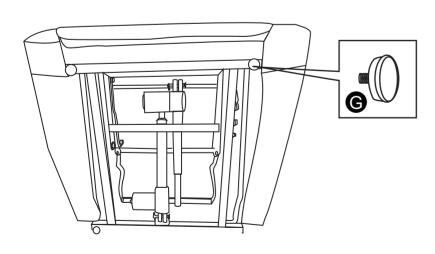


STEP 2

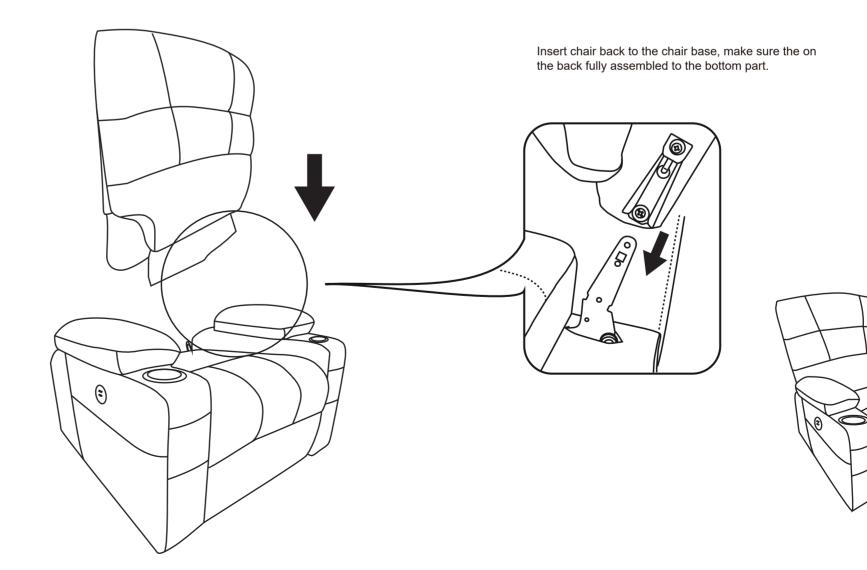

STEP 3

NOTE

- 1: Two same color wires connect.
- 2: Plug power cord and power source into outlet, then you can use all function.
- No.1 and No.2 wire connect with the backrest and chair base.
- No.3 wire connects with backrest and power cord(C).
- No.4 wire connects with backrest and remote control(D).
- No.5 wire connect the motor and the wire from lift control. No.6 wire connect the motor and power source (E).
- No.7 wire connect the USB charge ports and backrest.

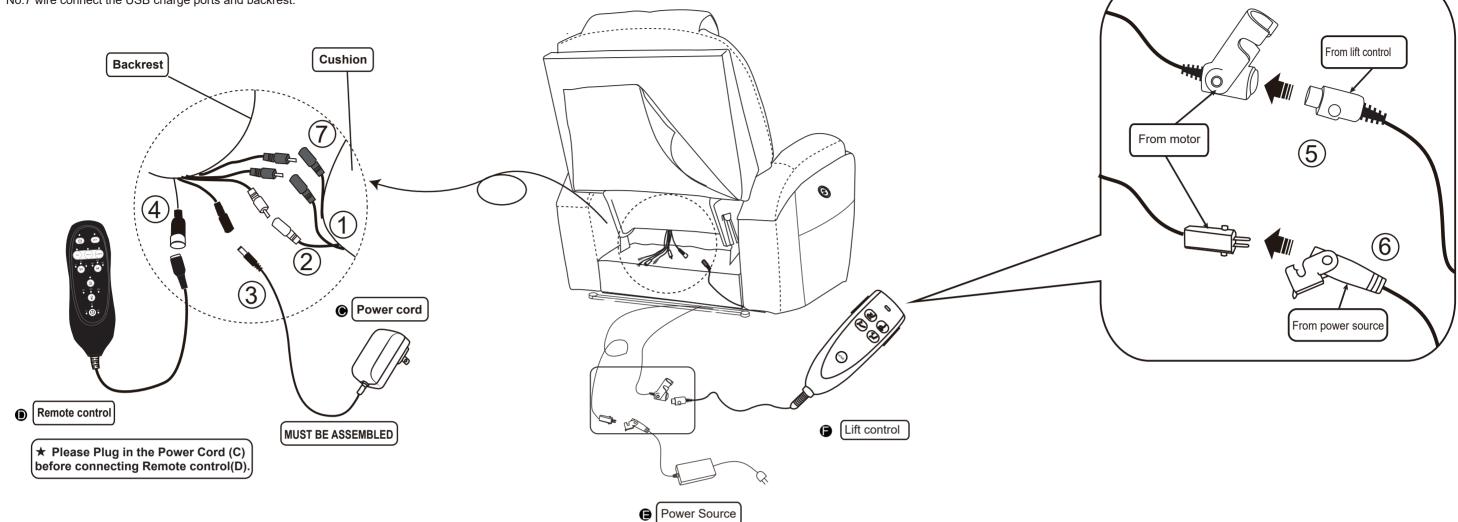



How to get footrest latch down and stand?

