

## Install on Toilet



1.Lay toilet rail around the commode, and adjust marbles of lower foot tube in the suitable height for user. Adjust marbles in horizontal bar for suitable width. Then fasten commode bowl by screwing up butterfly nuts of clamps.

2.Cross black safety bandage from back of toilet rail, then buckle the plastic fastener.

### **Warning**

Physically challenged users should be supervised or assisted whenever using this product.

Please ensure fully understand the warning as below before use the products.

Please ask agency or health specialist in case of any damage to body and rail.


Check fixed parts correctly and firmly fixed.

Any abrasion or invalid of the rail, please ask a professional to maintain or change.

Uneven and wet, slippery ground is not available. Clean regularly, wiping down with a soft cloth and a mild detergent/disinfectant solution.

Do not use corrosive solution and abrasive cleaners.

# MANUAL Toilet Safety Rail

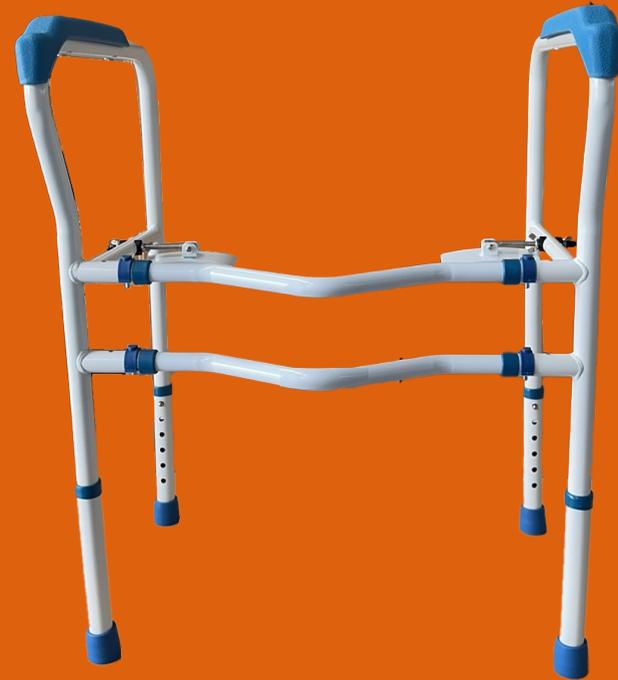

## Description

This product can be used for home health care and daily life. It does great favour in helping user standing up and sitting down.

Physically challenged users should be supervised or assisted whenever using this product.

### Main Feature

1. Max loading: 136kg
2. Main material: Carbon steel
3. Six level adjustable




## Before You Start

Please read all instructions carefully.  
Retain instructions for future reference.  
Separate and count all parts and hardware.  
Read through each step carefully and follow the proper order.

We recommend that, where possible, all items are assembled near to the area in which they will be placed in use, to avoid moving the product unnecessarily once assembled.

Always place the product on a flat, steady and stable surface.  
Keep all small parts and packaging materials for this product away from babies and children as they potentially pose a serious choking hazard.


## Product Configuration

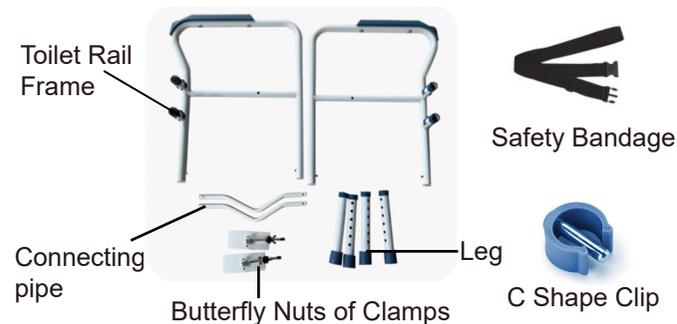

## Parts List

- Toilet Rail Frame x2
- Butterfly Nuts of Clamps x2
- Safety Bandage x1
- C Shape Clip x2
- Legs x4
- Connecting Pipes x2

## Installation Steps

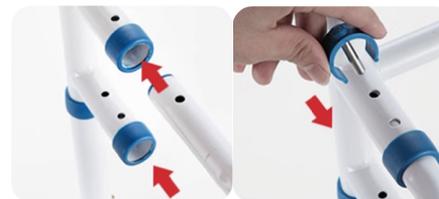

1. Pull two handgrips to opposite direction. When two holes in inner and outside tube coincide with each other, insert C shape clips in two sides.



2. Detach the small plug, nut and arc sheet from the adjustment shaft, then insert the shaft into the hole of handrail tube from outside. Put all the parts on the shaft again. Do the same in the other side.



3. Insert legs into frame. Make sure all the marbles in the same level and pop up.
4. Final installation effect.