ITEM #0116778/0116807/0116811/0116825


GRAB BAR - EXPOSED SCREW (18 IN., 24 IN., 36 IN., 42 IN.)

MODEL #B01B7100NP/B01B7201NP/B01B7300NP/B01B7301NP

Français p. 03
Español p. 05

PACKAGE CONTENTS

Part	Description	Quantity
A	Grab Bar	1
B	2 in. Stainless Steel Screw	6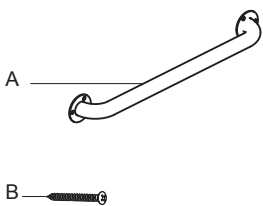

WARNINGS AND CAUTIONS


- To prevent serious or fatal injury: for use only by persons weighing less than 300 pounds. Consult healthcare professional for advice and instructions specific to your needs before use.
- Follow this installation instruction carefully. Proper installation is the installer's responsibility.
- Failure to follow correct installation procedure can cause the grab bar to become loose, which can result in serious injury.
- Grab bar should be mounted to secure surfaces. Wood blocking is preferable behind all surface.

IMPORTANT INFORMATION

- Grab Bars meet the American Disabilities Act (ADA) specifications tested to withstand 300 lbs. pull when securely installed per instructions.

PREPARATION

- Before beginning assembly, remove all parts from carton. Compare parts with parts list and diagram above. If any part is missing or damaged, do not attempt to mount or use the product. Contact customer service for replacement.
- Estimated Assembly Time: Approximately 10 minutes
- Tools Required for mounting: Phillips Screw Driver, Drill, 1/8 in. Drill Bit, Level, Pencil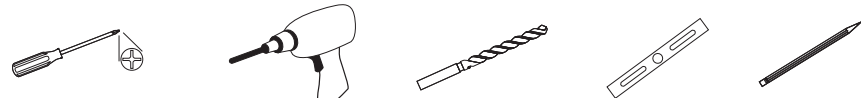

Questions, problems, missing parts? Before returning to your retailer, call our customer service department at 1-866-538-5535, 8 a.m. - 8 p.m., EST, Monday - Friday, 9 a.m. - 6 p.m., EST, Saturday.

1

INSTALLATION INSTRUCTIONS

Steps

- Place the grab bar on the wall where desired so that the mounting holes are positioned over stud. Use a level, if desired, to verify placement. Mark the 6 mounting hole locations with a pencil.
- Remove product from the wall and drill 1/8 in. holes at the 6 marked locations.
- Mount the grab bar using the supplied screws.

Note :
Grab bars are to be mounted to a secure surface. The manufacturer shall not accept responsibility for any and all damages caused by lack of wall strength or improper installation.

Fig. 1

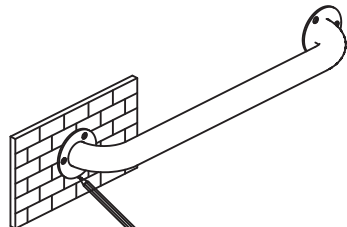
Fig. 2


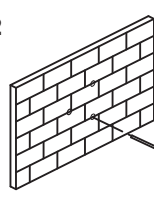
Fig. 3

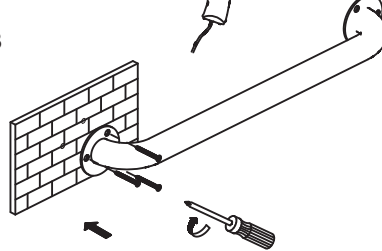CARE AND MAINTENANCE

Cleaning: To maintain and protect the fine luster of all products, clean with a soft damp cloth only. Do not use a detergent or cleanser that may harm the finish.

LIMITED LIFETIME WARRANTY

If this product fails due to a defect in materials or workmanship at any time during the life of the product, the manufacturer will replace it free of charge, postage-paid at their option. Simply contact customer service at 1-866-538-5535 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn. The manufacturer disclaims all other implied or expressed warranties including all warranties of merchantability and/or fitness for a particular purpose. As some states do not allow exclusions or limitations on an implied warranty, the above exclusions and limitations may not apply. This warranty gives you specific legal rights, and you may have other rights that vary from state to state.

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