


Thank you for your purchase!

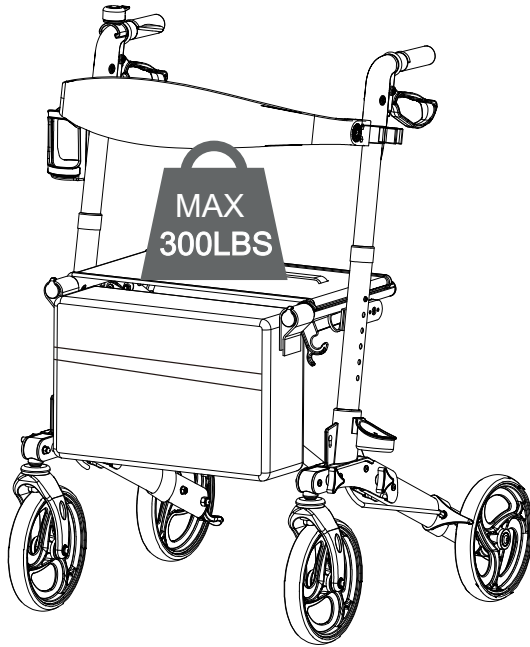

Medical Rollator Walker

ASSEMBLY INSTRUCTIONS

Before You Begin:

Please identify all component parts and hardware pieces required before you begin. Carefully remove all of the components from the packaging and set aside for assembly. Assemble on a soft surface to prevent scratching during assembly.

Caution:

Tighten all components securely before use. Failure to do so may result in personal injury.

DO NOT use any sharp objects to open plastic wrapped components as damage to product or components may result.

Wheels may slide on carpeted surfaces when turning or braking

Warning:

CHOKING HAZARD - Small Parts. Adult Assembly Required.

DO NOT ALLOW CHILDREN TO CLIMB ON THE WALKER.

Customer Service Email: CustomerExperience@theubiquegroup.com | Phone: 866-552-2810

196861160325



Congratulations!

You just purchased a piece of happiness. At Ubique Group, we believe you can transform your spaces into the most inspired places with furnishings that are the perfect combination of form, function and style. As original innovators of online furniture retail, we offer the best selection of popular, contemporary styles at cool, comfortable prices. And, with decades of proven shipping and packaging expertise, our Extended Quality Checklist ensures a rewarding, hassle-free experience time and time again. It's just what you need to bring your vision to life.

Thank you & Enjoy!!
your friends at Ubique Group



LIKE IT!

*Share it on social
& Tell a friend!
#Spaces2Places*



LOVE IT!

*Write an online
product review
where purchased*



NOT SURE?

*Let us help!
Contact our
customer support.*

Have a question, concern or simply need a hand? Your satisfaction is our top priority. Contact us at 866-552-2810 or customerexperience@theubiquegroup.com. We're here to help!

Browse Our Branded Collections

There's something for everyone inside our family of brands.

FLASH
FURNITURE®

MERRICK | LANE

Taylor & Logan

Emma+Oliver


MASADA
RUGS


HBCY
CREATIONS

BlackArc®

Visit www.theubiquegroup.com/brands or scan this code.





We pride ourselves on providing an excellent customer experience starting right here with your package and instructions. Everything you need should be right where you need it, so you can get to enjoying your new item ASAP. But despite our best efforts, sometimes "slips happen." If you need any support, from replacement parts to recommendations, please reach out to our wonderful customer support team at 866-552-2810 or customerexperience@theubiquegroup.com. We're here to help!

EVERY UBIQUE GROUP PRODUCT




Quality &
Durability Tested



Packaged
with Care



Shipped from
our U.S. Facility

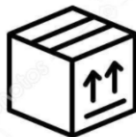

Designed
for Easy
Assembly






Ethically
Sourced


Please visit <https://www.theubiquegroup.com/warranty> for complete warranty information.

Read this manual before using this product. Failure to follow the instructions and safety precautions in this manual can result in serious injury or death. Keep this manual in a safe location for future reference.

PARTS:

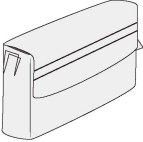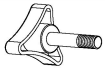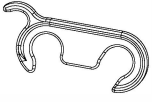
1		x1 Storage Bag	4		x1 Small Bell	7		x4 Triangle Knob
2		x1 Backrest Strap	5		x1 Crutch Clamp	8		x1 Main Frame
3		x1 Cup Holder	6		x1 Crutch Box	9		x1 Buckle

STEP 1

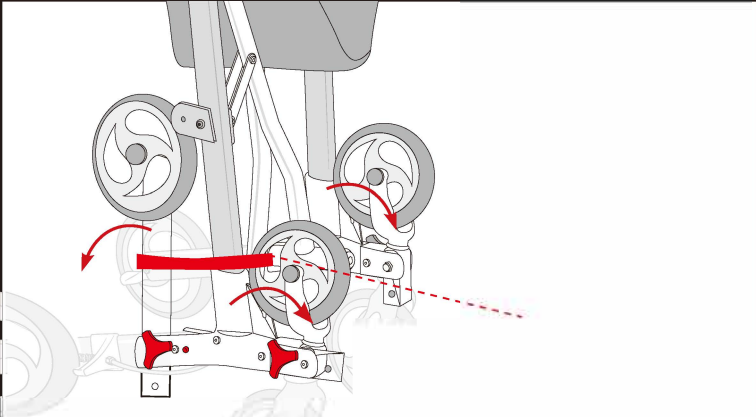


Remove the triangle knobs from both sides and the strap then flip the front and rear wheels downwards until they align with the installation holes.

STEP 2

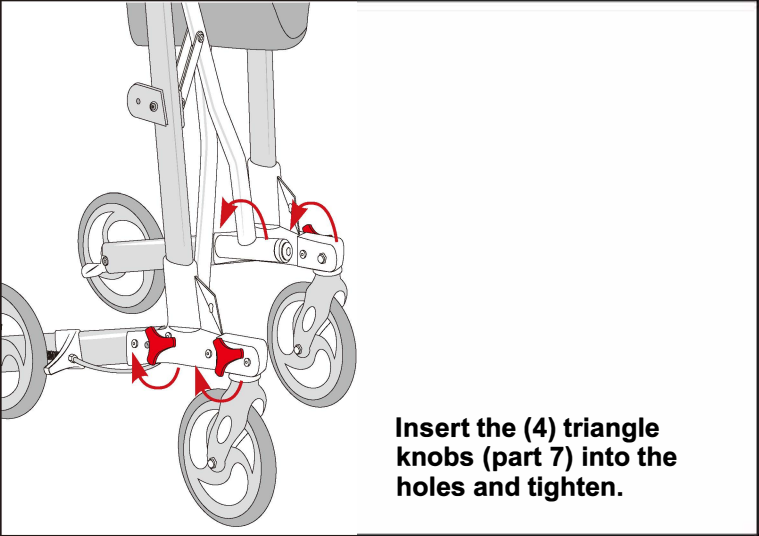

Insert the (4) triangle knobs (part 7) into the holes and tighten.


STEP 3

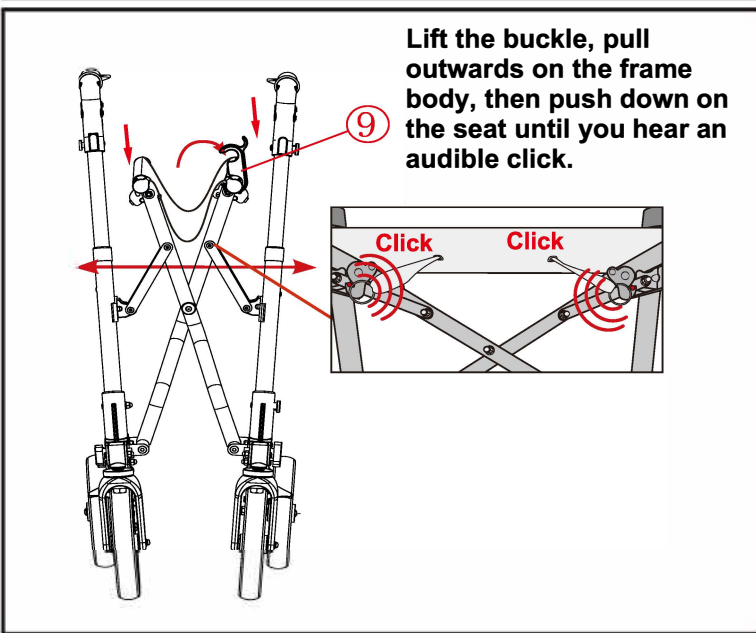

Lift the buckle, pull outwards on the frame body, then push down on the seat until you hear an audible click.

Click Click


STEP 4

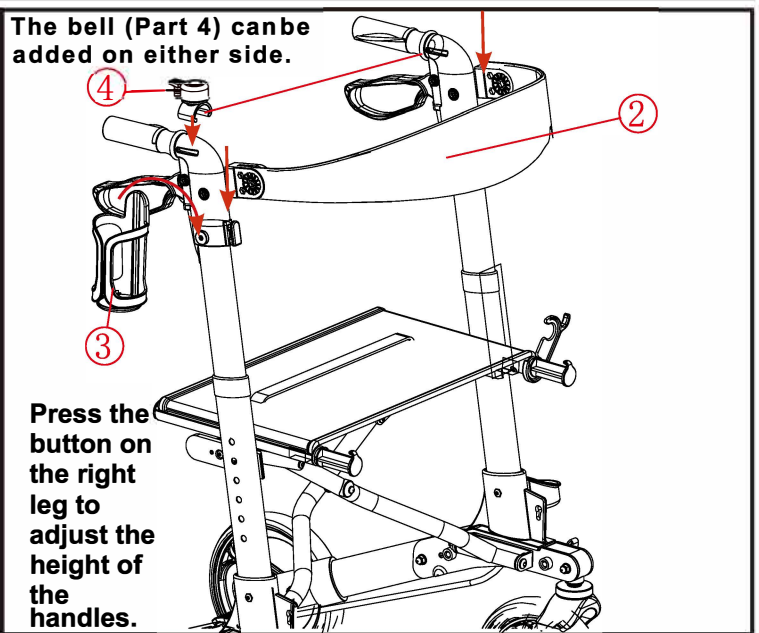


The bell (Part 4) can be added on either side.

Press the button on the right leg to adjust the height of the handles.


STEP 5

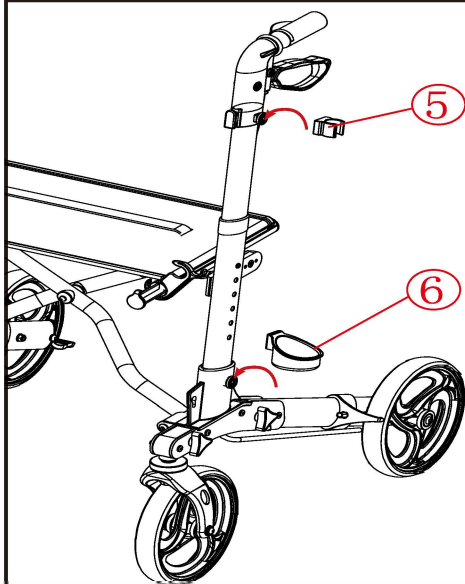

STEP 6

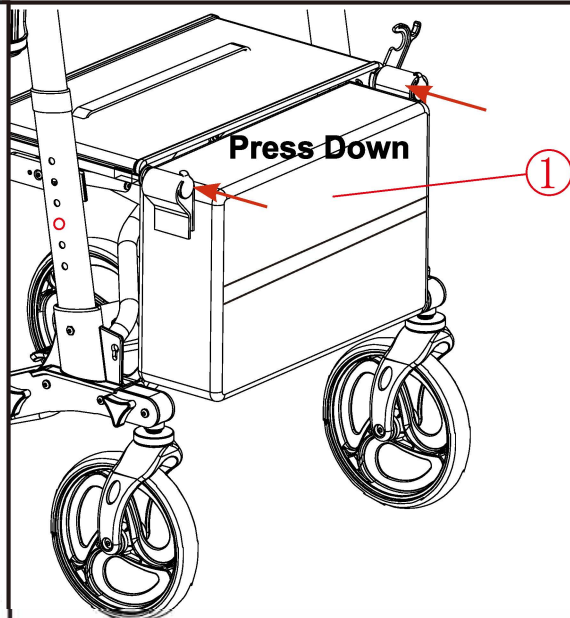


STEP 7

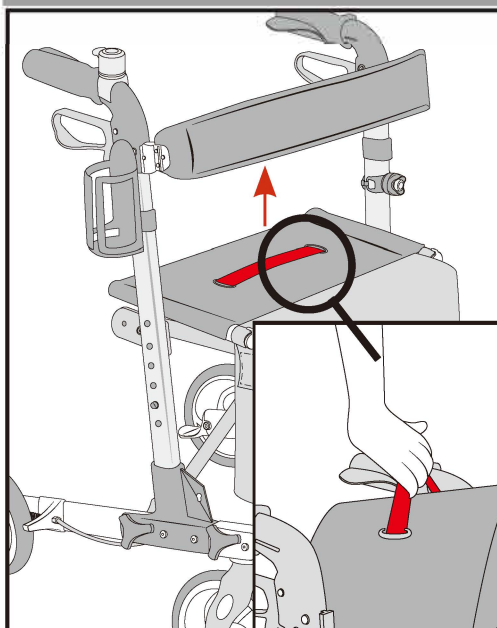


Pull up on the seat handle and press inwards on the frame body to fold.

STEP 8

**Pull
Brake
Handles
Up To
Slow The
Walker**

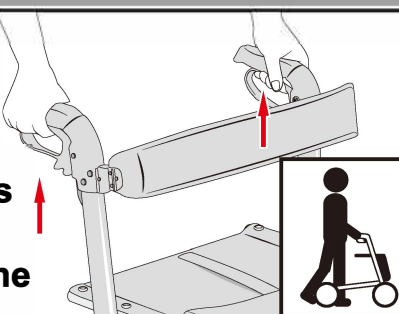

**Push Brake
Handles
Down to Lock
Rear Wheels**

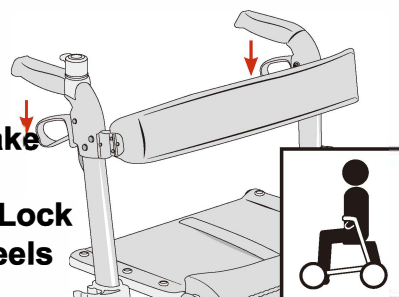

Quality and Safety Guidelines

- Please follow the static and dynamic weight capacity limits provided on this item. If you are unsure of the weight capacities for this product, email us at customerexperience@theubiquegroup.com or call 866-552-2810.
- Do not use the walker under the influence of alcohol, medication or other drugs that affect your physical or mental abilities.
- All wheels must have constant contact with the ground to ensure correct alignment of the walker.
- Do not use the walker on uneven surfaces or surfaces that may hinder the wheels such as sand or gravel.
- Do not use the walker on icy or wet surfaces or in reduced visibility conditions.
- Do not use the walker on stairs or escalators.
- Transport things solely in the bag provided, do not hang anything from the frame.
- Do not use the device as a walker while seated. Before using the seat, the parking brake must be activated.
- Consult your physician to determine the proper adjustment and use of this device.
- Make sure that the handles on both sides are set on the same height and are locked correctly.
- Regularly check all parts to ensure there is no damage to the grips, wheels, or frame and that all screws are securely tightened.
- No modification may be made to this device. Use only original parts supplied by the manufacturer to replace any broken or defective parts.
- Do not use the product if it is damaged!
- Hold firmly to the handles while using this product and move forward with short steps to ensure proper balance.
- In case of falling, let the walker fall to the side to avoid contact with your body.
- While using this product, beware of any indentations, holes, obstacles, or other ground deformities in the path of the walker.
- This product is not intended for use as a step stool, lever, or for moving furniture.
- This item is intended to seat one person at a time.
- DO NOT store outdoors or in a vehicle and protect from extreme temperatures (Over 100°F or Below 32°F).
- Ensure all appendages are clear of the interior foldable areas while folding or unfolding the walker.

INTENDED USE:

Walking aids are used as part of a rehabilitation program when the user is recovering from an injury or surgery or as a long-term aid when the user has a permanent condition that causes difficulty with walking. This device helps maintain balance and aids in the prevention of injuries (particularly falls) allowing individuals to independently attend to their needs. Mobility aids with wheels may be used indoors or outdoors. Mobility aids are used to modify weight distribution so some of the weight carried through the legs while walking is transferred to the arms of the walker frame for support. The product is intended for adult use.

Do Not Use If You Have Any Of The Following:

Severe Cognitive Disorder
Severe Loss of Balance
Severe Motor or Perception Disorders
Weakness in the Upper Limbs

Please review and retain all assembly instructions, safety and quality guidelines and product warnings for future reference.