


Thank you for your purchase!

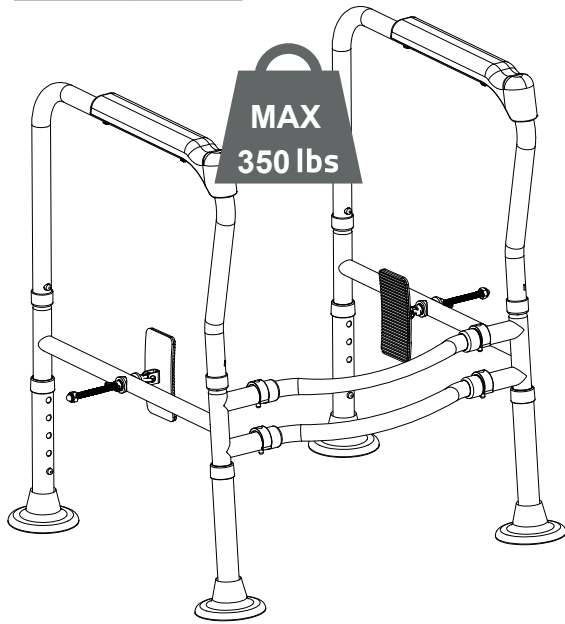

# Adjustable Toilet Safety Frame

## ASSEMBLY INSTRUCTIONS

### Before You Begin:

Please identify all component parts and hardware pieces required before you begin. Carefully remove all of the components from the packaging and set aside for assembly. Assemble on a soft surface to prevent scratching during assembly.

### Caution:

Tighten all components securely before use. Failure to do so may result in personal injury.

DO NOT use any sharp objects to open plastic wrapped components as damage to product or components may result.

### Warning:

CHOKING HAZARD - Small Parts. Adult Assembly Required.

This adjustable toilet frame is not intended to be used as a walker and should only be used with a toilet.

DO NOT ALLOW CHILDREN TO CLIMB ON THE TOILET ARMREST



# Congratulations!

You just purchased a piece of happiness. At Ubique Group, we believe you can transform your spaces into the most inspired places with furnishings that are the perfect combination of form, function and style. As original innovators of online furniture retail, we offer the best selection of popular, contemporary styles at cool, comfortable prices. And, with decades of proven shipping and packaging expertise, our Extended Quality Checklist ensures a rewarding, hassle-free experience time and time again. It's just what you need to bring your vision to life.

**Thank you & Enjoy!!**  
*your friends at Ubique Group*



## LIKE IT!

*Share it on social  
& Tell a friend!  
#Spaces2Places*



## LOVE IT!

*Write an online  
product review  
where purchased*



## NOT SURE?

*Let us help!  
Contact our  
customer support.*

Have a question, concern or simply need a hand? Your satisfaction is our top priority. Contact us at 866-552-2810 or [customerexperience@theubiquegroup.com](mailto:customerexperience@theubiquegroup.com). We're here to help!

## Browse Our Branded Collections

There's something for everyone inside our family of brands.



MERRICK | LANE

Taylor & Logan

Emma+Oliver®



BlackArc®

Visit [www.theubiquegroup.com/brands](http://www.theubiquegroup.com/brands) or scan this code.





We pride ourselves on providing an excellent customer experience starting right here with your package and instructions. Everything you need should be right where you need it, so you can get to enjoying your new item ASAP. But despite our best efforts, sometimes "slips happen." If you need any support, from replacement parts to recommendations, please reach out to our wonderful customer support team at 866-552-2810 or [customerexperience@theubiquegroup.com](mailto:customerexperience@theubiquegroup.com). We're here to help!

## EVERY UBIQUE GROUP PRODUCT




Quality &  
Durability Tested



Packaged  
with Care



Shipped from  
our U.S. Facility

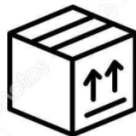

Designed  
for Easy  
Assembly







Ethically  
Sourced


Please visit <https://www.theubiquegroup.com/warranty> for complete warranty information.

**Read this manual before using this product. Failure to follow the instructions and safety precautions in this manual can result in serious injury or death. Keep this manual in a safe location for future reference.**


PARTS:
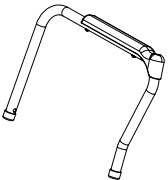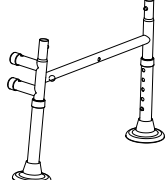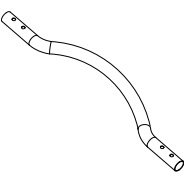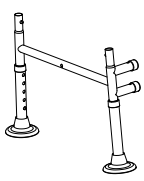
|   |                                                                                   |                        |   |                                                                                   |                         |   |                                                                                     |                    |
|---|-----------------------------------------------------------------------------------|------------------------|---|-----------------------------------------------------------------------------------|-------------------------|---|-------------------------------------------------------------------------------------|--------------------|
| 1 |  | x2 Armrest             | 4 |  | x1 Right Frame with Leg | 7 |  | x4 E- Type Plug    |
| 2 |  | x2 Connector           | 5 |  | x2 Butterfly Nut        | 8 |  | x2 Adjustable Grip |
| 3 |  | x1 Left Frame with Leg | 6 |  | x2 Nylon Cap Nut        | 9 |  | x2 Curved Gasket   |


STEP1

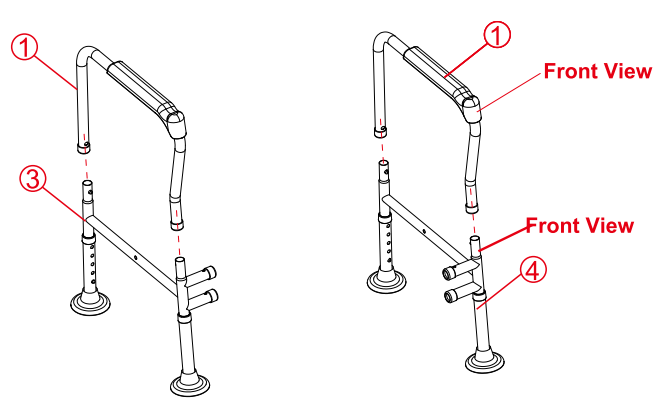


STEP2

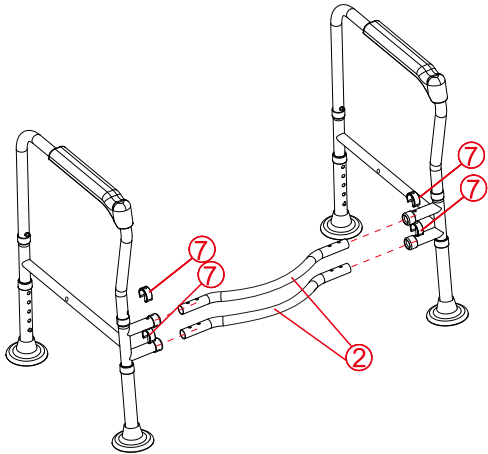

STEP3

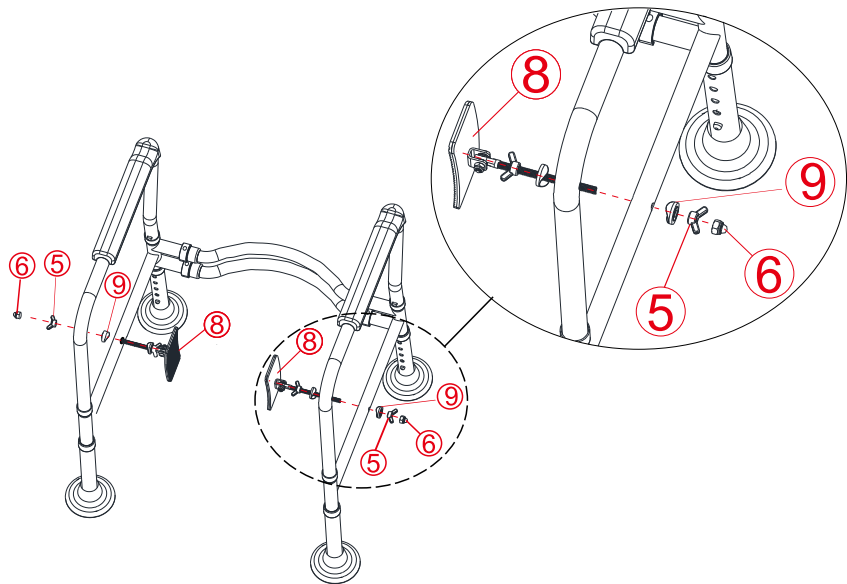

STEP4

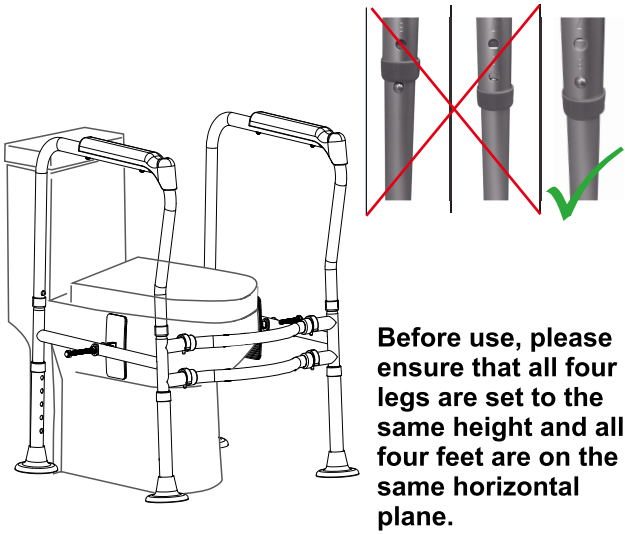

## Quality and Safety Guidelines

- **Please review and retain all assembly instructions, safety and quality guidelines and product warnings for future reference.**
- Please follow the static and dynamic weight capacity limits provided on this item. If you are unsure of the weight capacities for this product, email us at [customerexperience@theubiquegroup.com](mailto:customerexperience@theubiquegroup.com) or call 866-552-2810.
- Please use this item on a stable, level surface only. Do not use this item if any instability is detected.
- Ensure frame clamps are firmly attached and stable before use.
- This item is intended for sitting only, do not climb or stand on this item.
- Please take care to keep the frame stationary when lowering yourself into a seated position.
- Hold onto both handrails when lowering or rising from the toilet seat to avoid instability.
- Please seek assistance during use if you have limited function in your arms or legs.
- This item was designed for one person at a time.
- Ensure frame clamps are fully retracted before attempting to remove from the existing toilet.
- This item is intended for indoor use only.
- Store indoors and keep away from moisture. For additional storage instructions, please email us at [customerexperience@theubiquegroup.com](mailto:customerexperience@theubiquegroup.com) or call 866-552-2810.
- Please inspect this product regularly to make sure there are no damaged or worn parts. Also inspect to make sure all hardware is tightened correctly and securely. For inspection instructions, please email us at [customerexperience@theubiquegroup.com](mailto:customerexperience@theubiquegroup.com) or call 866-552-2810.
- If any parts are damaged or worn, please stop using the product. Repair this product with manufacturer supplied parts only.
- Please visit [www.theubiquegroup.com](http://www.theubiquegroup.com) for our product warranty details or email us at [customerexperience@theubiquegroup.com](mailto:customerexperience@theubiquegroup.com) or call 866-552-2810.

**Please make sure to get additional assistance from care giver for use if you have any of the following:**

Severe Cognitive Disorder  
Severe Loss of Balance  
Severe Motor or Perception Disorders  
Weakness in the Upper Limbs

TO REGISTER FOR THE WARRANTY, SCAN THE QR CODE BELOW:

