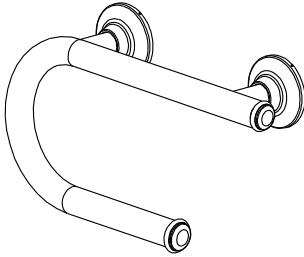
Designer Grab Bar with Integrated Toilet Paper Holder Instruction Sheet

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Moen Home Care is a leader in grab bars, offering top-quality products that are attractive and innovative, with performance features like a 250lb. weight capacity.

The entire Moen Home Care family of products offers people new options for independent living by providing outstanding comfort, convenience and style.

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Note: Not to be used by persons weighing more than 250lbs.

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Integrated accessories are not designed to support weight leverage.

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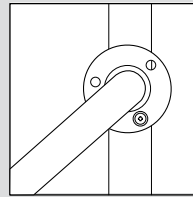
All Moen Home Care grab bars feature the exclusive patented SecureMount™ mounting flange.

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Conventional Mounting / Montage conventionnel

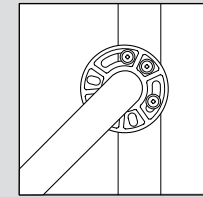
Conventional mounting configurations often only allow for one or two screw connections to the stud, making installation difficult and less secure.

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SecureMount™ / SecureMount™ / SecureMount™

The patented SecureMount™ design makes grab bar installation easier and more secure with an innovative design that features multiple slots for screws, allowing secure connection with three screws into the wood stud at any angle.

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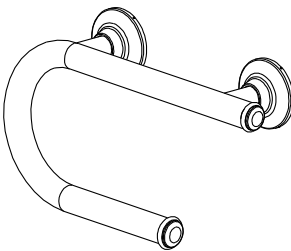
Components included / Pièces incluses / Componentes incluidos

Grab bar, with SecureMount™ mounting flanges and decorative covers

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Requires a 3 1/2" space behind the wall and works in the following substrates

- 1/2" or 5/8" Dry Wall
- 1/2" or 5/8" Dry Wall with Tile
- 1/2" or 5/8" Dry Wall with Marble
- 1/8" or 1/4" Fiberglass Tub Surround

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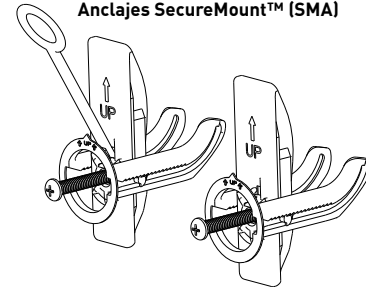
Set screw size:
M5x.8 x 5mm long
(2 extra set screws included)

Extra screws
Vis
supplémentaires
Tornillos
adicionales

2.5 mm Hex wrench (allen wrench)
Clé hexagonale (clé Allen)
Llave hexagonal (llave Allen)

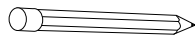
Tools Required / Outils requis / Herramientas necesarias

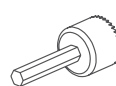
Drill
Perceuse
Taladro

Level
Niveau
Nivel

Pencil
Crayon
Lápiz

1 1/2" Hole Saw (carbide tipped hole saw for tile)
Scie cloche de 1 1/2 po (lame à pointe en carbure pour découper la tuile)
Sierra perforadora de 1-1/2 pulg (con punta de carburo para sierra perforadora de azulejos)

These instructions are for installation using provided SecureMount™ Anchors (Drywall mounting – off stud location)

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1

Tape template to wall at desired installation location. Use level to check horizontal alignment. Template included in hardware pack.

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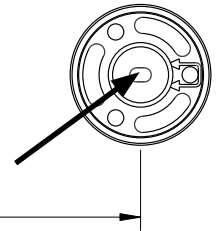
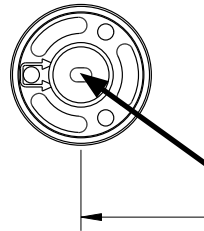
Warning: Use stud finder to avoid drilling into studs.

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2

Use pencil to mark center hole of SMA flanges.

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3

Use 1-1/4" hole saw and drill hole at marked locations.

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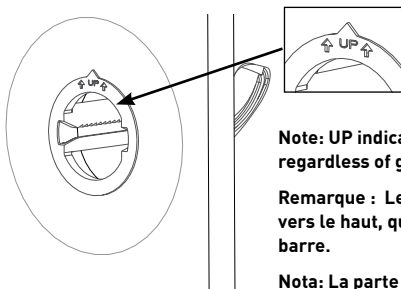
4

Insert SMA guide into drilled hole as illustrated.

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Note: UP indicator must always point up regardless of grab bar orientation.

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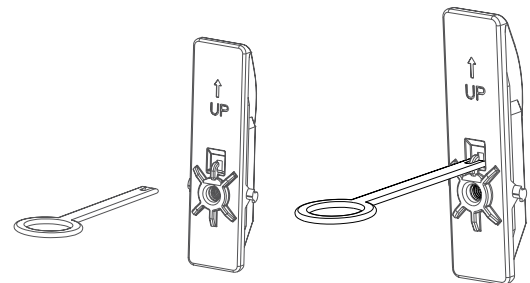
5

Connect pull tab to SecureMount™ Anchor.

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Conecte la lengüeta de halar al Anclaje SecureMount™.

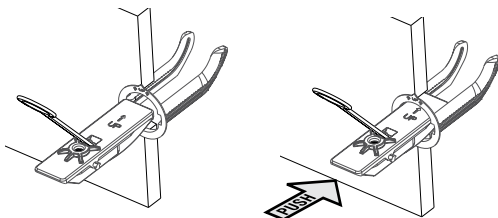
6

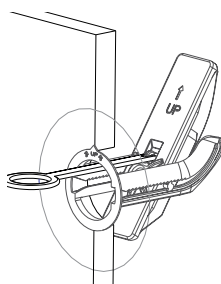
With pull tab up, insert the anchor all the way into the hole in the wall.

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7

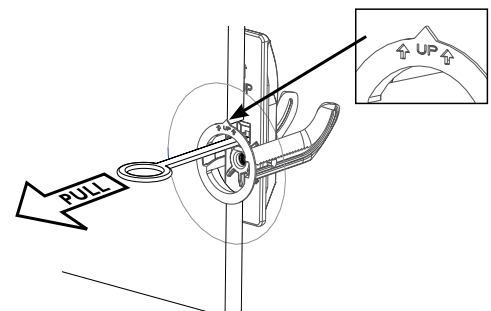
8

Pull back on the pull tab and disconnect it from the anchor.

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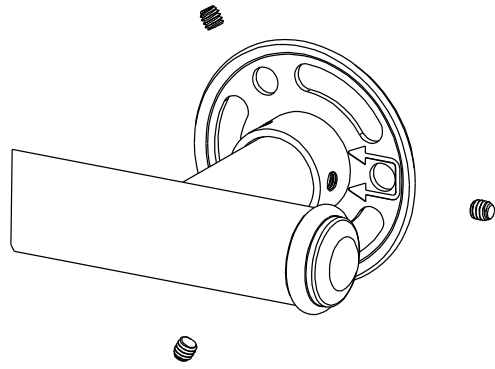
9

Disassemble bar from SecureMount™ flange by removing set screws.

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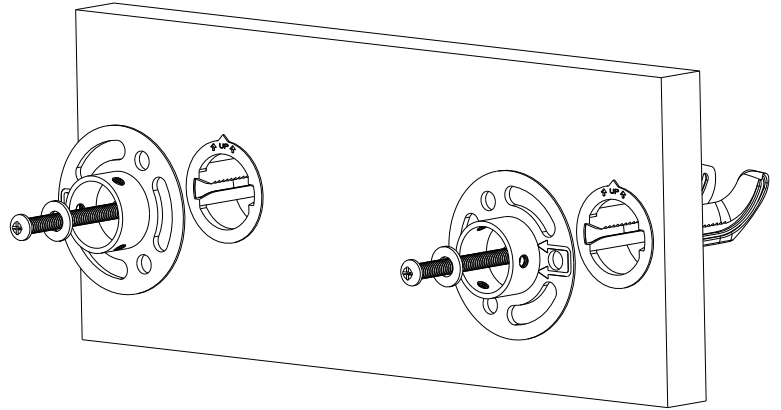
10

Install SecureMount™ flange. Insure the stamped arrows are pointing inward as illustrated.

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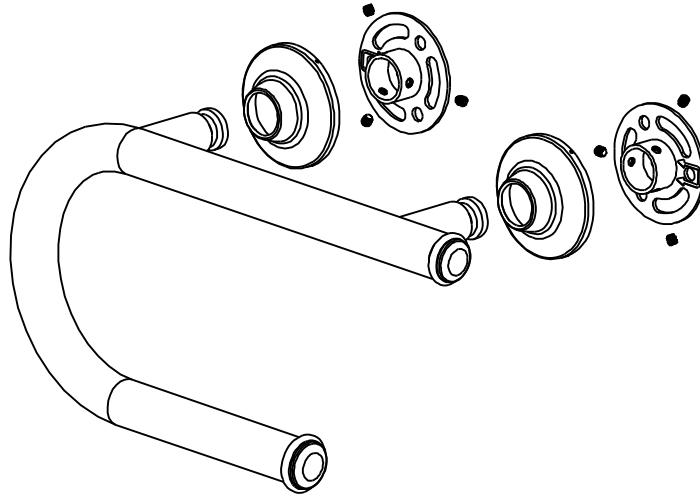
11

Slide decorative cover onto grab bar and install the bar onto the SecureMount™ flanges by tightening the screws.

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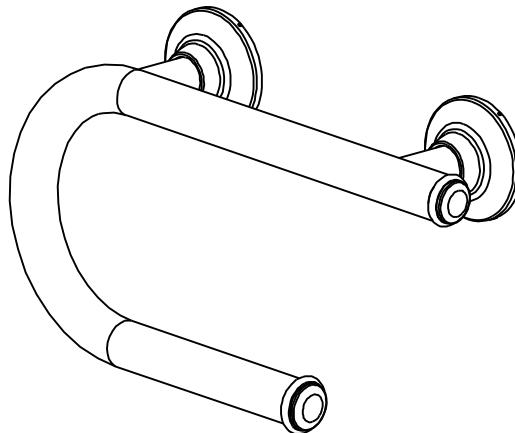
Encliquetez les caches.

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WARNING:

TO PREVENT SERIOUS OR FATAL INJURY: FOR USE ONLY BY PERSONS WEIGHING LESS THAN 250 POUNDS. Consult a healthcare professional for advice and instructions specific to your needs before use. Contact CSI immediately at 1-800-882-0116 if any components are missing or damaged. Install in accordance with CSI's instructions. Do not modify this product in any way. Check grab bar periodically after installation to be sure all screws are secure. FOR QUESTIONS OR ASSISTANCE WITH INSTALLATION CONTACT CSI AT 1-800-882-0116.

LIMITED LIFETIME WARRANTY:

If this product fails due to materials or workmanship at any time during the life of the product, Creative Specialties International (CSI), a division of Moen Incorporated, will replace it free of charge, postage-paid. Simply contact CSI at 1-800-882-0116 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn. TO THE EXTENT PERMITTED BY LAW, CSI DISCLAIMS ALL OTHER IMPLIED OR EXPRESS WARRANTIES INCLUDING ALL WARRANTIES OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE.

LIMITATION OF LIABILITY:

THIS WARRANTY IS YOUR SOLE REMEDY AND CSI SHALL NOT BE LIABLE FOR ANY DAMAGES, WHETHER DIRECT, INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL, EXEMPLARY, OR OTHERWISE, INCLUDING LOST REVENUES AND LOST PROFITS, ARISING OUT OF ANY THEORY OR RECOVERY, INCLUDING STATUTORY, CONTRACT OR TORT. NOTWITHSTANDING THE TERM OF ANY LIMITED OR IMPLIED WARRANTY, OR IN THE EVENT THAT ANY LIMITED WARRANTY FAILS OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL CSI'S ENTIRE LIABILITY EXCEED THE PURCHASE PRICE OF THIS PRODUCT. SOME STATES AND PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU. THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE RIGHTS WHICH VARY FROM STATE TO STATE AND PROVINCE TO PROVINCE.

For additional warranty information, visit www.moen.com/homecare

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Home Care and Creative Specialties International are registered trademarks of Moen Incorporated ©2011
www.moen.com/homecare

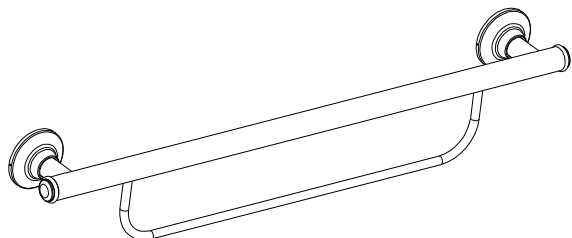
Creative Specialties is a division of Moen Incorporated. © 2011
Creative Specialties est une division de Moen incorporée. © 2011
Creative Specialties es una división de Moen Incorporated. © 2011

Designer Grab Bar with Integrated Towel Bar Instruction Sheet

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The entire Moen Home family of products offers people new options for independent living by providing outstanding comfort, convenience and style.

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Integrated accessories are not designed to support weight leverage

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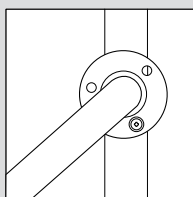
Conventional Mounting / Montage conventionnel / Montaje convencional

Conventional mounting configurations often only allow for one or two screw connections to the stud, making installation difficult and less

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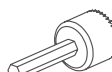
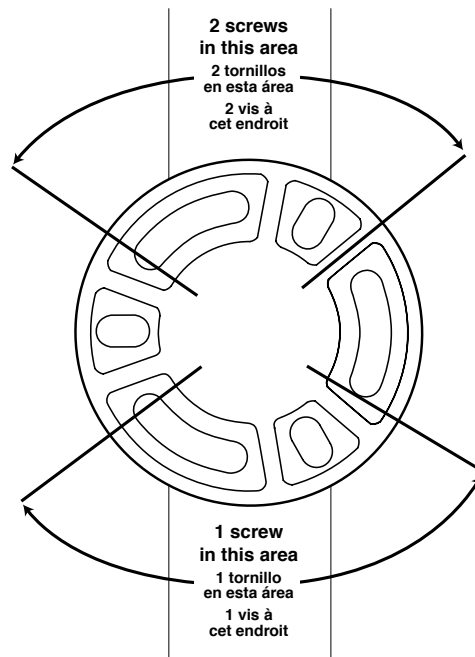
Preferred installation is into wood backing (studs) on both the left and right sides of the grab bar. One SecureMount anchor is included in case a stud is not available on one side of the grab bar.

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1

Determine desired mounting location on the wall, ensuring at least one side is located over a stud.
Locate the stud behind the wall by using a stud sensor or other stud locating method.

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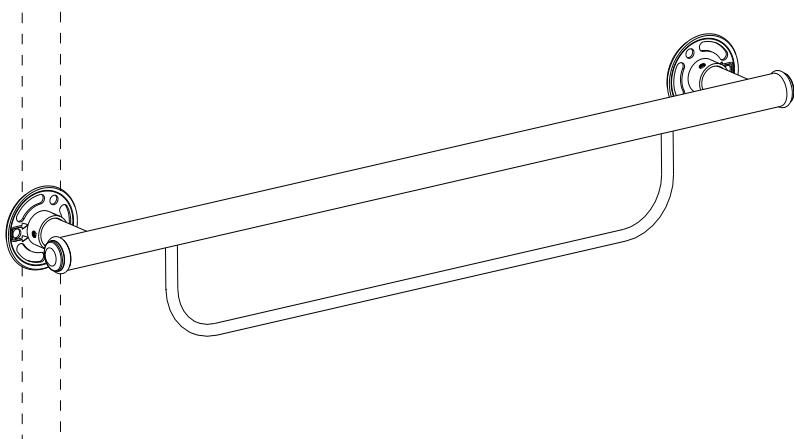
Determine the location of the desired mounting on the wall, ensuring that at least one (1) side remains installed over a column.

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2

Place the bar on the wall; ensuring one mounting flange is located on a stud. Use a level to make sure the bar is level on the wall.

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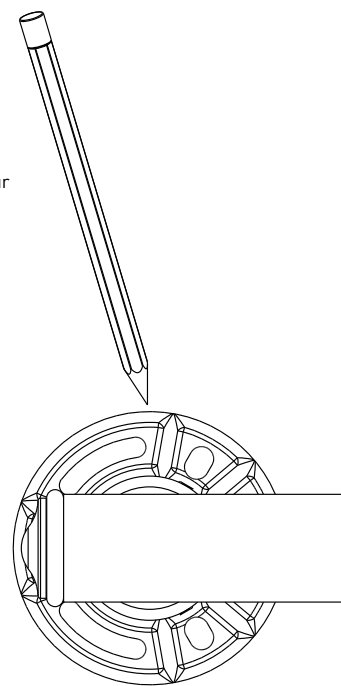
3

Trace both flange perimeters with pencil.

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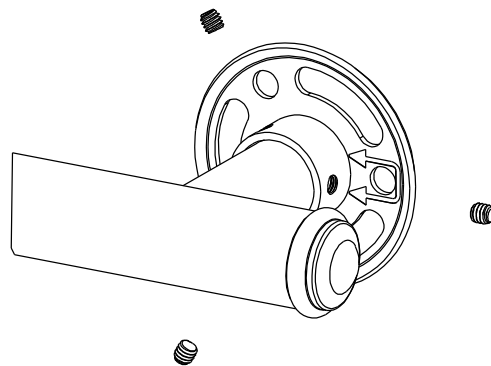
4

Remove 3 set screws from each flange and disassemble bar from SecureMount™ flanges.

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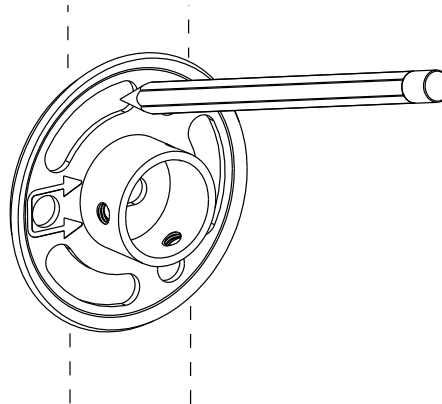
5

On the stud side mounting location, center the flange in the traced circle and mark 3 mounting locations as illustrated. (Refer to figure 1). Insure the stamped arrows are pointing inward as illustrated.

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6

Drill 1/8 diameter pilot holes at each screw location. Make sure to drill into the stud. Refer to fig 1 for hole locations.

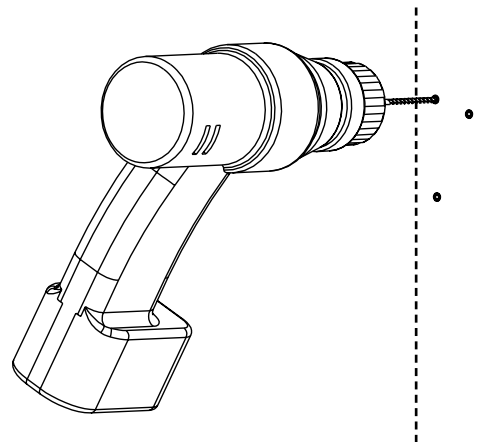
NOTE: For easier screw installation, drill the pilot hole length of the screw.

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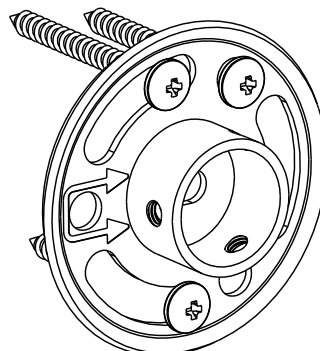
7

On the stud mount side, attach the secure mount flange to the wall using the 3 screws and tighten until secure.

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8

If the second mounting location is over a stud, repeat steps 6-8 and go step 15. If a stud is not available, use the provided SMA and continue to step 9.

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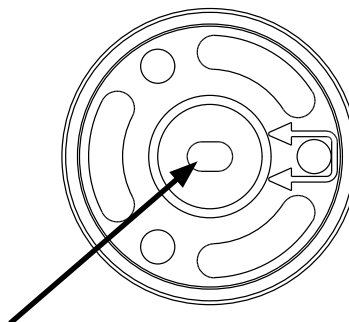
9

Center 2nd Secure Mount flange in 2nd traced circle and use pencil to mark the center hole. Use 1 1/4" hole saw and drill hole at marked location.

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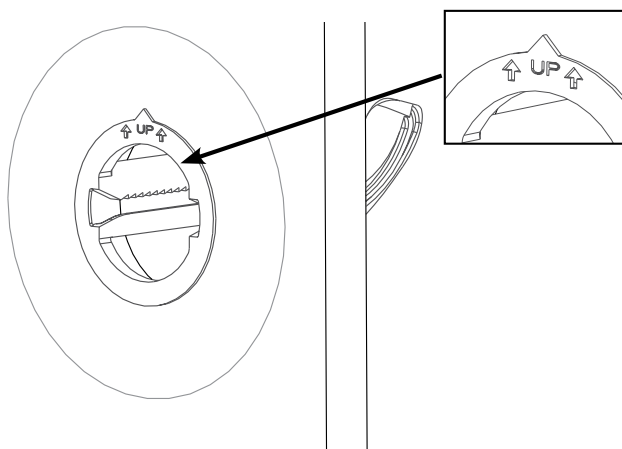
11

Connect pull tab to SecureMount™ anchor.

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