

Product instruction manual

Dual-function nursing bed

Seracare

1. Tips



1.1 This product will be shipped in 2 packages. They may not arrive at the same time. Please wait.

1.2 Thank you for your purchase! We are committed to providing you with high quality products. In the unlikely event that your product is damaged, missing parts or has problems with assembly, please contact our customer service team.


We will offer you the best solution as soon as possible.

In order to solve your problem quickly, please provide us with some pictures that clearly show the product problem. Thanks !

1.3 Please keep all small parts out of the reach of children when assembling. May be fatal if swallowed or inhaled. Children should not climb or play on the product to avoid serious injury from furniture tipping over. To avoid possible injury from movement, keep all parts and packaging (film, plastic bags, foam, etc.) away from infants and children.

1.4 Please match the corresponding parts and accessories to the part numbers in the manual. If parts and accessories do not match, check the information in this manual. When assembling, first align all screws loosely with the corresponding pre-drilled holes and then tighten the screws one by one. During product use, periodically check the tightness of screws and tighten as necessary to ensure structural integrity. Please keep sharp and corrosive objects away from the product to avoid damage or injury.

1.5 It is recommended to install it with two people.

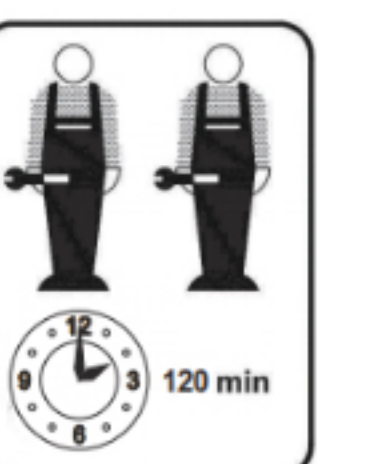

2. Product overview

2.1 Composition of product structure

The dual-function electric nursing bed consists of a headboard, a footboard, side panels, foot bars, armrests, a motor and its drive system, and a handheld electric controller.

2.2 Product model and specification: KYG-D-02.

2.3 Product functions

- 1) Raise or lower the legs
- 2) Raise or lower the back
- 3) Make the legs and back rise or fall in linkage

3. Product performance parameters

3.1 Dimension parameter table

Project	Size (in)
Bed length	78.74
Bed width	39.37
Bed height	34.25
Bed surface height	16.93

3.2 Electrical parameters

- 1) Input voltage: AC 1 00V~240V ; Operating voltage: DC 35V ;
- 2) Frequency: 50/ 60Hz ;


3.3 Performance parameters

- 2.3.1 Folding angle of the electric bed backrest: 0°~70° (±5°) .
- 2.3.2 Folding angle of the electric bed's leg section: 0°~30° (±5°) .

4. Product operating conditions







- 1) Ambient temperature: +5°C~+40°C;
- 2) Relative humidity: <90%;
- 3) Atmospheric pressure: 860hPa~1060hPa

5. Instructions for using the remote control

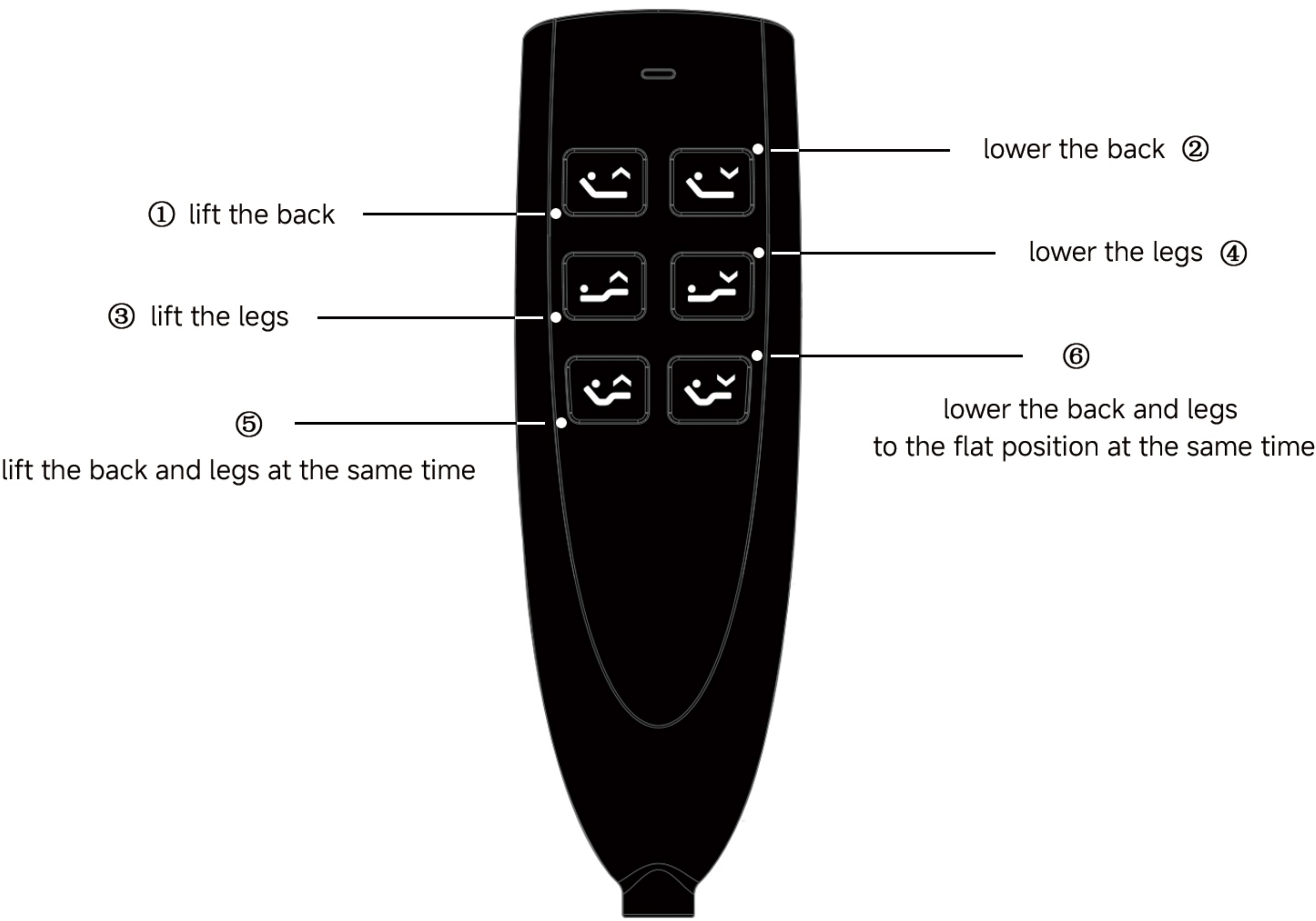

- ① Press and hold the button to raise the back. back lifting angle: 0-70°(±5°).
- ② Press and hold the button to lower the back.
- ③ Press and hold the button to lift the legs, which helps promote blood circulation.
leg lifting angle: 0-30°(±5°).
- ④ Press and hold the button to lower the legs.
- ⑤ Press and hold the button to lift the back and legs at the same time.
- ⑥ Press and hold the button to lower the back and legs to the flat position at the same time.

6. List of accessories

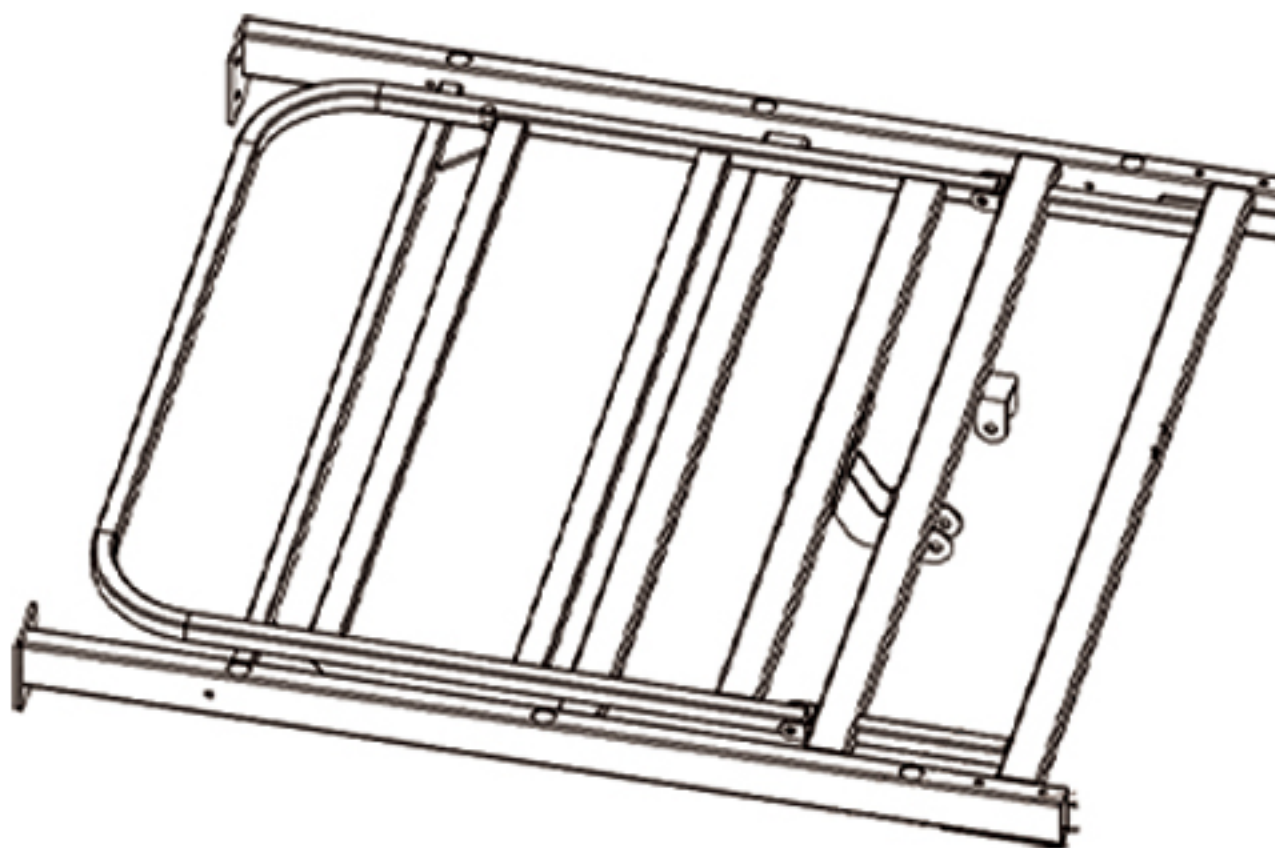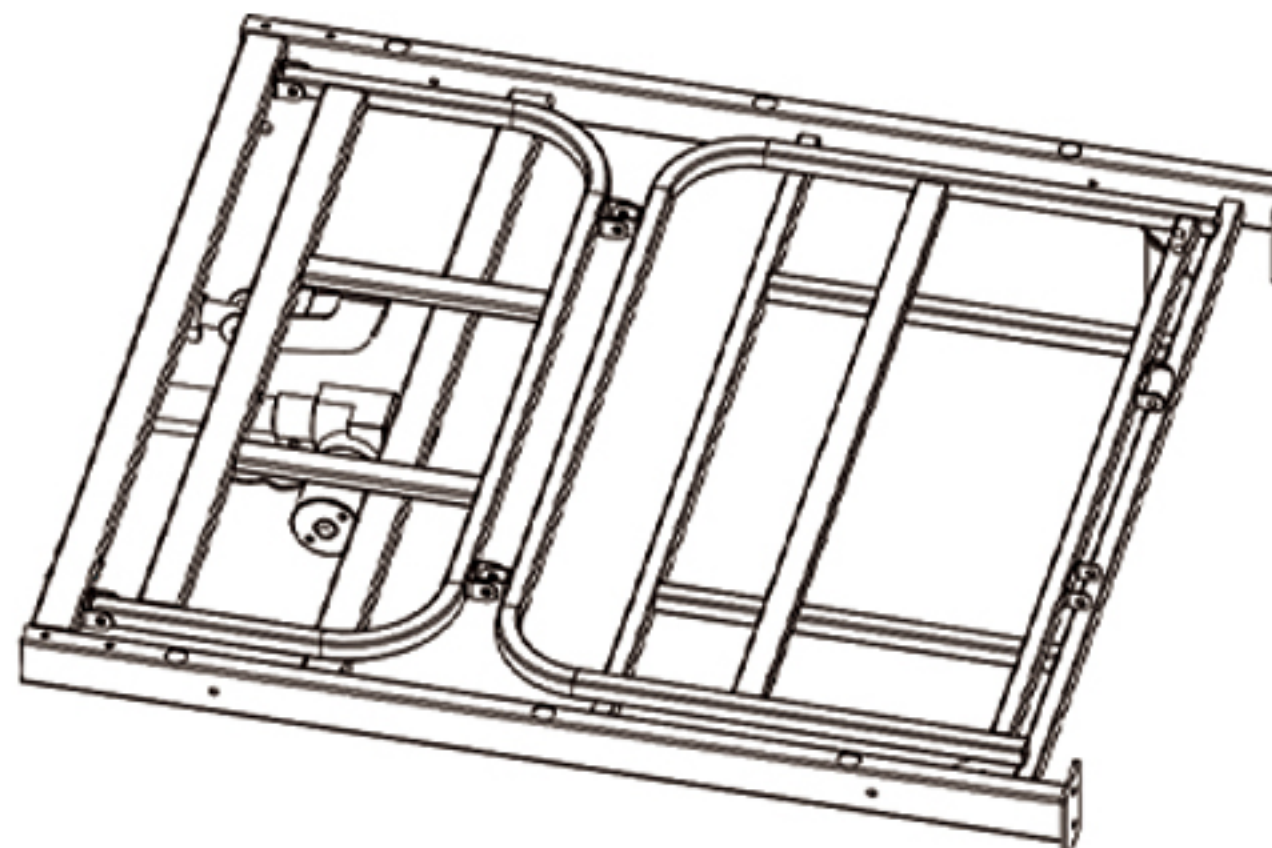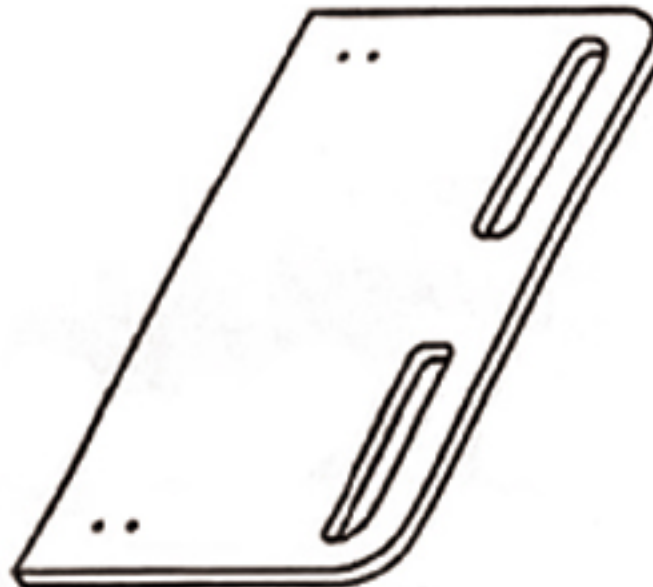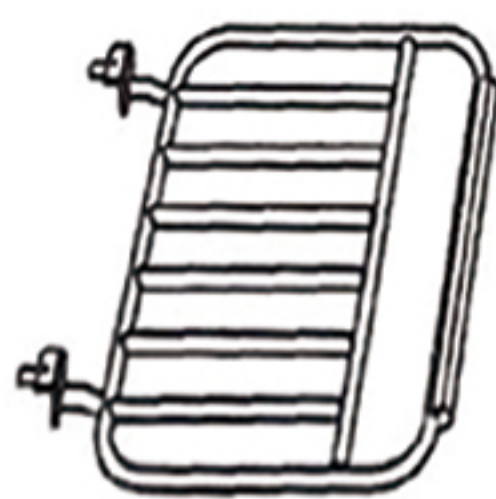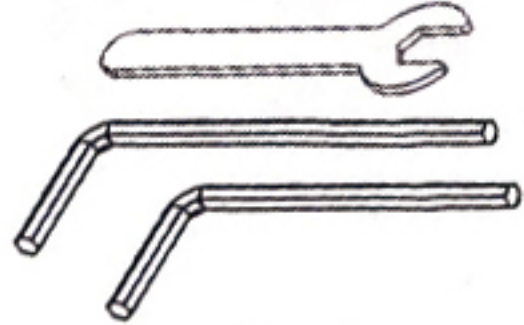
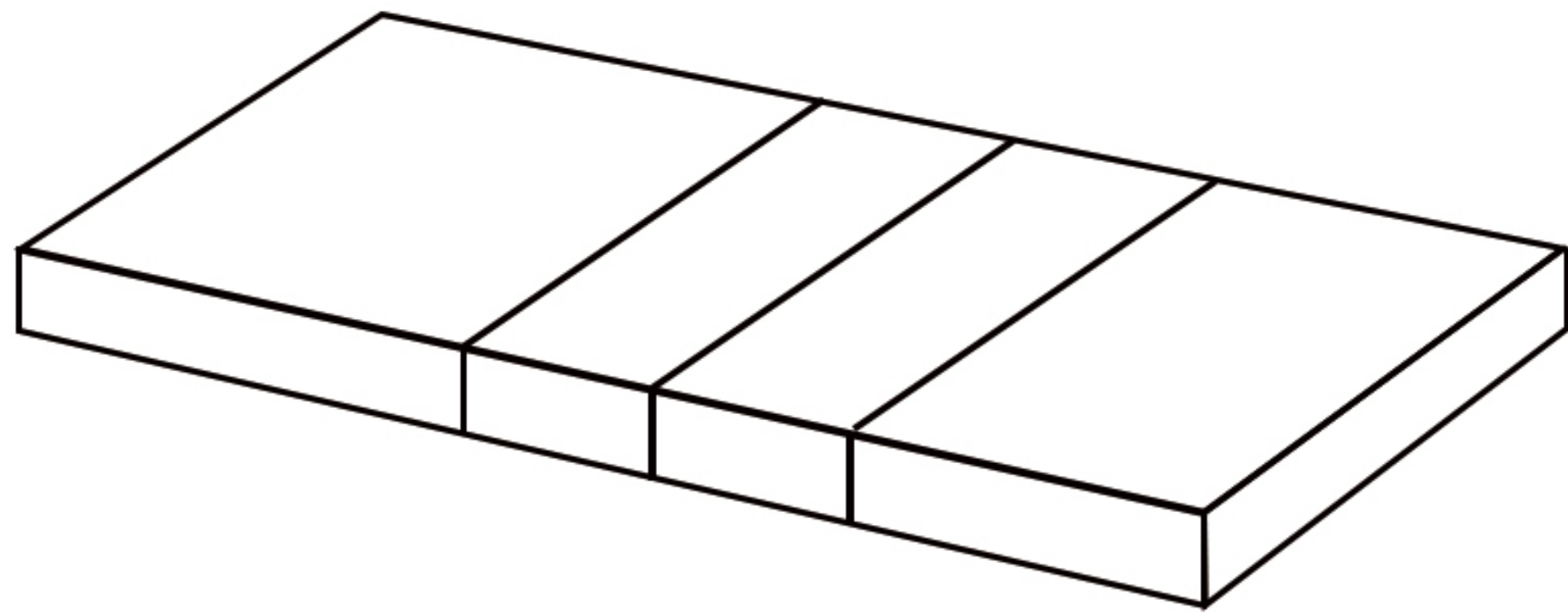
① x 1		② x 1			
					
③ x 2	④ x 2	⑤ x 8	⑥ x 8	⑦ x 2	⑧ x 2
 Connector	 Fixings	 Screw: M6*60mm	 Nut: M6	 Clevis Bolt: M10*50mm	 R Type Cotter Pin
⑨ x 4	⑩ x 4	⑪ x 4	⑫ x 8	⑬ x 2	
	 Screw: M10*30mm		 Screw: M6*35mm		
⑭ x 8	⑮ x 8	⑯ x 2		⑰ x 2	
 Screw(A): M6	 Screw(a): M6*12mm			 Tool	
⑱ x 1					
					

7. Installation steps (A)



①	Bed frame (left)	1PCS
②	Bed frame (right)	1PCS
③	Connector	2PCS
④	Fixings	2PCS
⑤	Screw : M6*60mm	8PCS
⑥	Nut: M6	8PCS

⑦	Clevis Bolt : M10*50mm	2PCS
⑧	R Type Cotter Pin	2PCS

7.1 Installation steps (B)

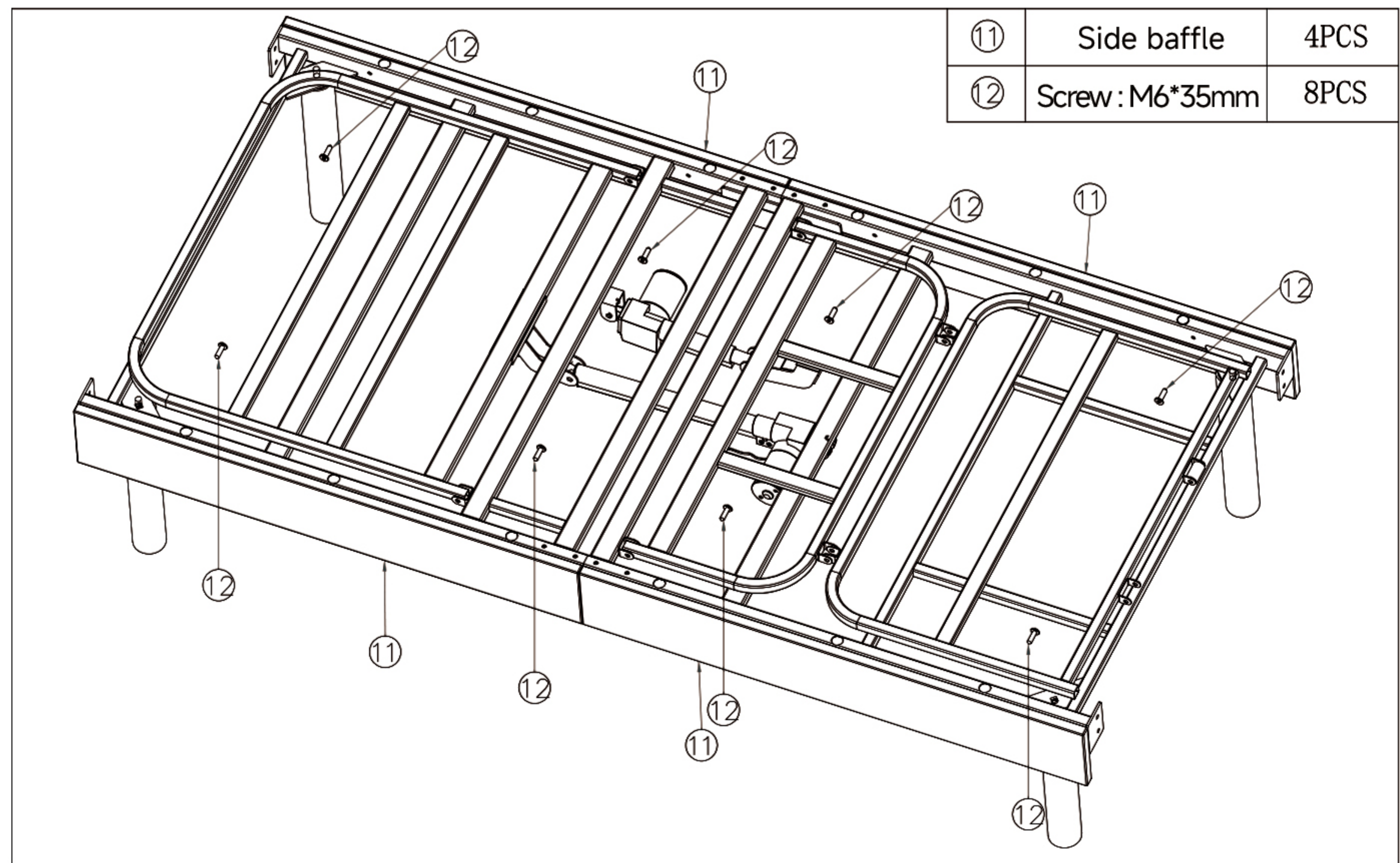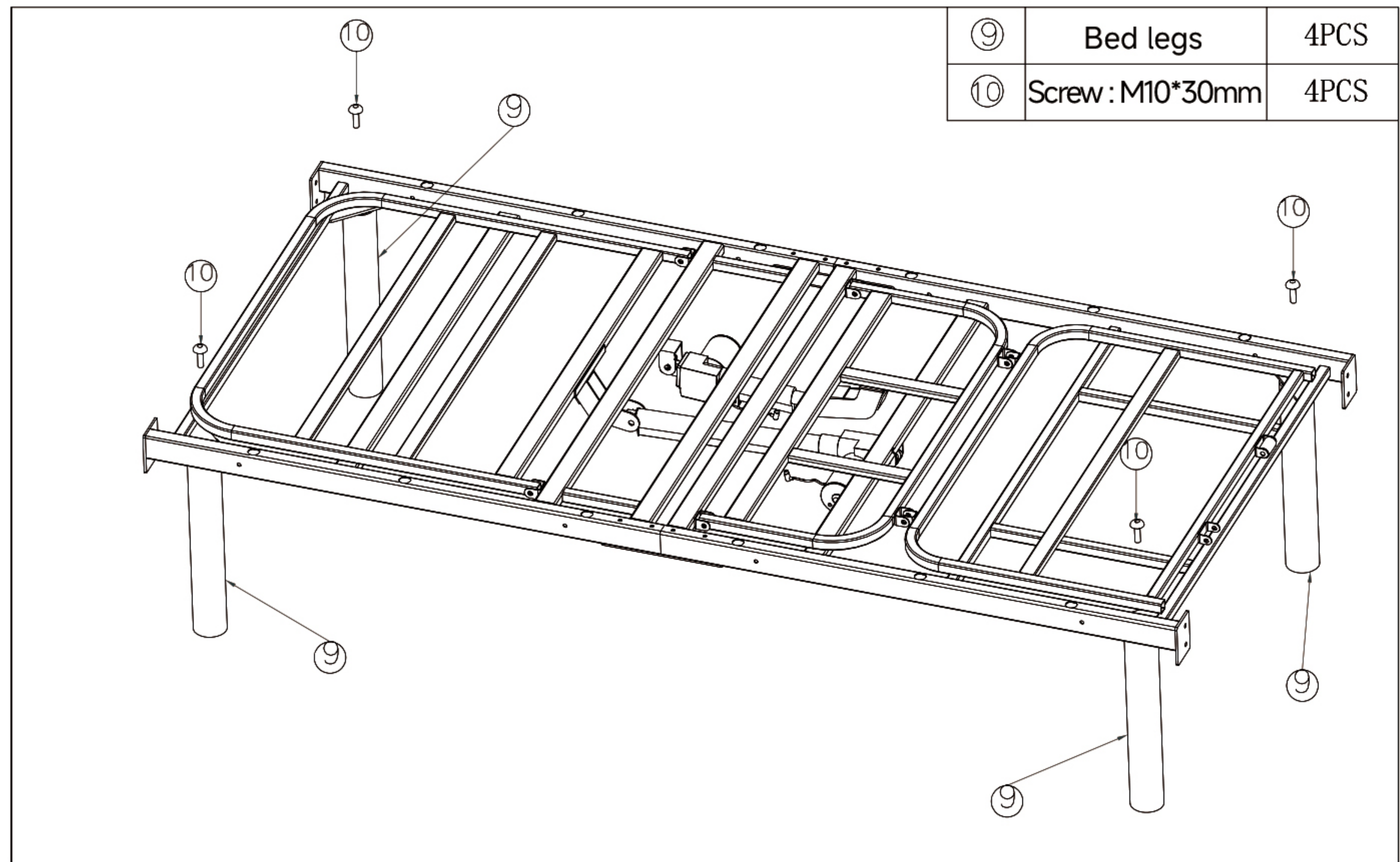

7.2 Installation steps (C)

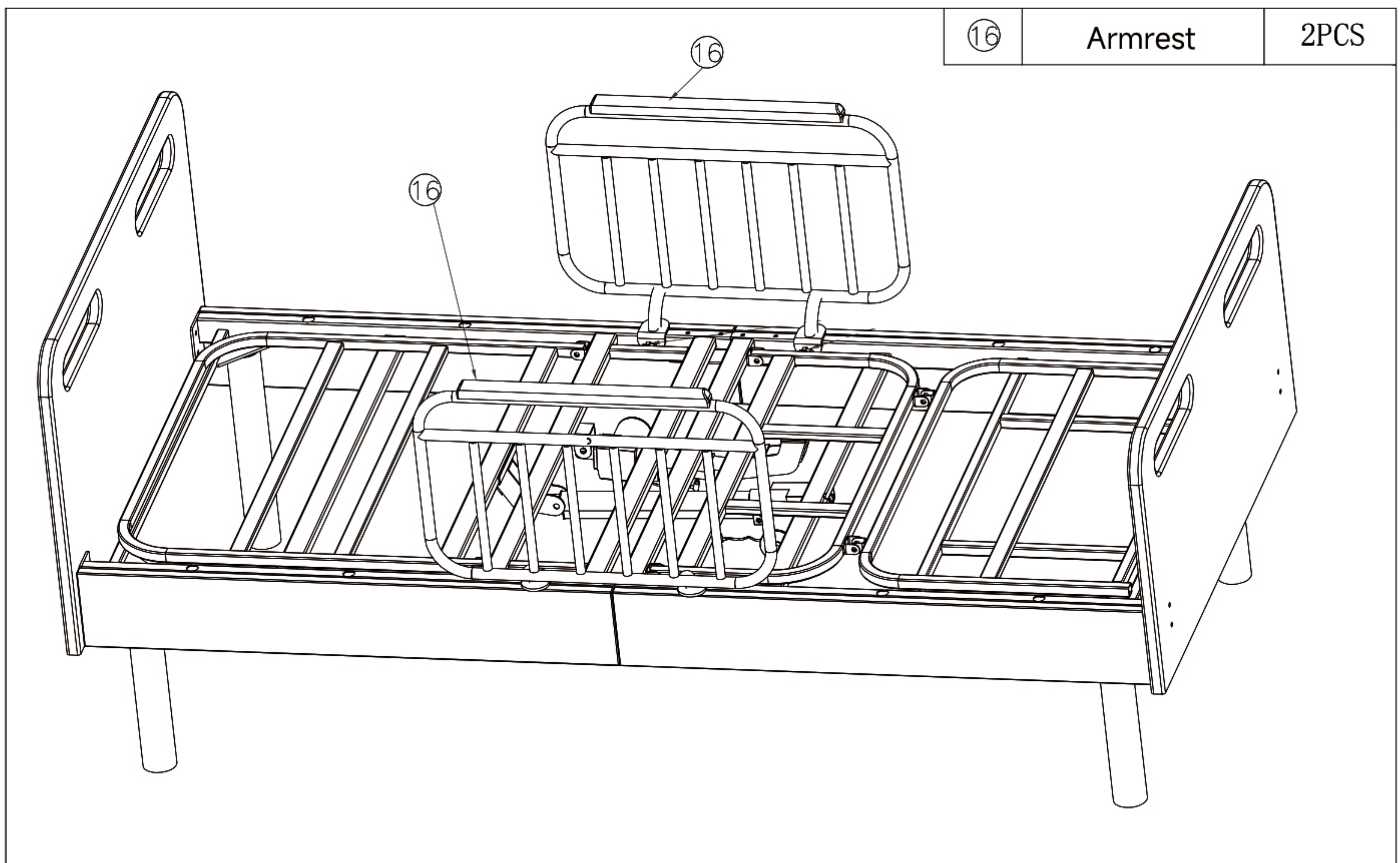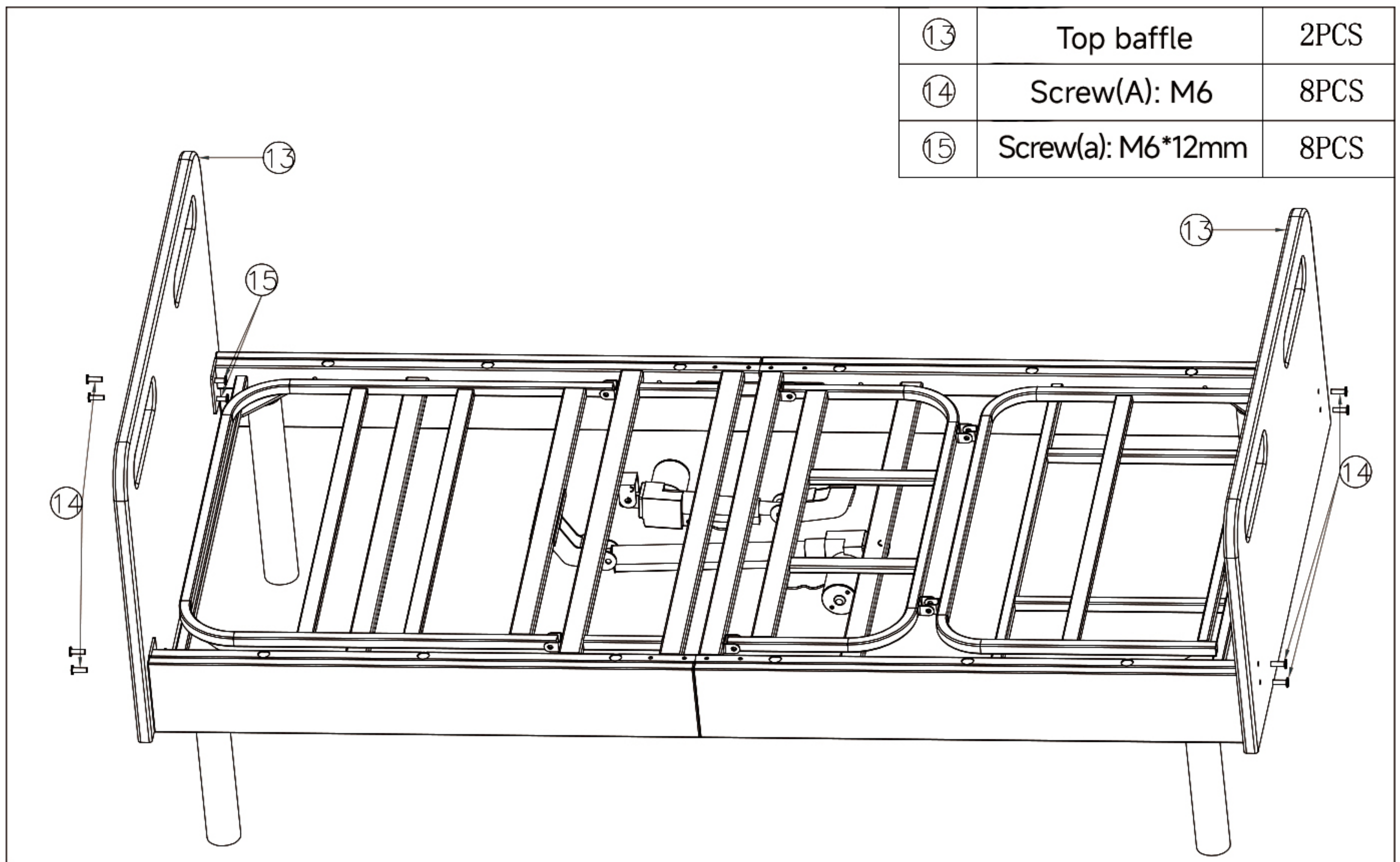

7.3 Installation steps (D)

18	Mattres	1PCS
----	---------	------

