LIFT CHAIR: lift mechanism pushes the entire chair up to help the senior stand up easily without adding stress to back or knees





FITTING LIST

				Contraction of the second seco			
CHAIR SEAT A	1	CHAIR BACK B	1	TRANSFORMER C	1	POWER CABLE D	1
SUPPORTING FOOT E	2	SCREW F	4	GASKET G	4	REMOTE CONTROL H	1

• Step 2: Assembly the two Supporting Feet (E) on the bottom of mechanism



• STEP 3: Attach Chair Back (B) to Chair Seat (A) by aligning the metal grooves on backrest to brackets on the main frame as shown. Ensure the Chair Back (B) slides down fully over the brackets on the Chair Seat (A).



• STEP 4: Connect the Transformer (C) with the Power Cable (D) and connect the Remote Control (H) with the power cable in the side pocket.

