Safe-er-Grip™

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INSTRUCTIONS FOR USE:

Easy to Install:

- With latches up, position Safe-er-Grip™ where desired on tub or shower wall.
- Apply pressure to the handle while flipping the latches down.
- Be sure to attach to smooth, dry, non-porous surfaces only. If installed on tile, the tile must be a 4" x 4" or greater. Do not install on tile grout lines. Not to be used on painted surfaces (ie: drywall).

IMPORTANT:

Check Safe-er-Grip regularly to ensure it is still firmly attached. If it can be moved or feels at all loose, flip up the latches and reinstall as per instructions above.

Easy to Remove:

- Release both latches.
- Slide a fingertip under rubber seal at each end to release suction. If necessary, slide something flat, such as a credit card, under the suction cups to release grip.

CAUTION: Safe-er-Grip is to be used as a balance assist only. Not for body weight leverage – body weight is not to be applied to product. (If device is required to bear any body weight, or is used for leverage, it is recommended that a traditional permanently mounted grab bar be installed.) Check attachment before each use and reattach if loose.

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