


Thank you for your purchase!

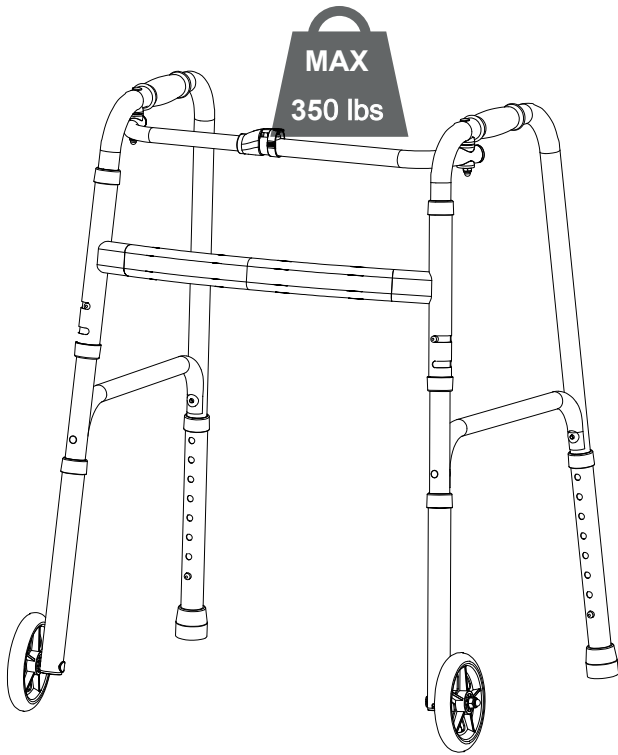

## **Adjustable Walker with Fixed or Wheeled Front Legs**

### **ASSEMBLY INSTRUCTIONS**

#### **Before You Begin:**

Please identify all component parts and hardware pieces required before you begin. Carefully remove all of the components from the packaging and set aside for assembly. Assemble on a soft surface to prevent scratching during assembly.

#### **Caution:**

Tighten all components securely before use. Failure to do so may result in personal injury.

DO NOT use any sharp objects to open plastic wrapped components as damage to product or components may result.

#### **Warning!**

Please take care to operate and use the folding adjustable walker as intended. Please do not use the top or middle front bars as handholds and please ensure that all height adjustment holes on the legs are set at the same level.

CHOKING HAZARD - Small Parts. Adult Assembly Required.

DO NOT ALLOW CHILDREN TO CLIMB ON THE WALKER

---

**Customer Service Email: [CustomerExperience@theubiquegroup.com](mailto:CustomerExperience@theubiquegroup.com) | Phone: 866-552-2810**

196861160363



# Congratulations!

You just purchased a piece of happiness. At Ubique Group, we believe you can transform your spaces into the most inspired places with furnishings that are the perfect combination of form, function and style. As original innovators of online furniture retail, we offer the best selection of popular, contemporary styles at cool, comfortable prices. And, with decades of proven shipping and packaging expertise, our Extended Quality Checklist ensures a rewarding, hassle-free experience time and time again. It's just what you need to bring your vision to life.

**Thank you & Enjoy!!**  
*your friends at Ubique Group*



## LIKE IT!

*Share it on social  
& Tell a friend!  
#Spaces2Places*



## LOVE IT!

*Write an online  
product review  
where purchased*



## NOT SURE?

*Let us help!  
Contact our  
customer support.*

Have a question, concern or simply need a hand? Your satisfaction is our top priority. Contact us at 866-552-2810 or [customerexperience@theubiquegroup.com](mailto:customerexperience@theubiquegroup.com). We're here to help!

## Browse Our Branded Collections

There's something for everyone inside our family of brands.



MERRICK | LANE

Taylor & Logan

Emma+Oliver®



BlackArc®

Visit [www.theubiquegroup.com/brands](http://www.theubiquegroup.com/brands) or scan this code.





We pride ourselves on providing an excellent customer experience starting right here with your package and instructions. Everything you need should be right where you need it, so you can get to enjoying your new item ASAP. But despite our best efforts, sometimes "slips happen." If you need any support, from replacement parts to recommendations, please reach out to our wonderful customer support team at 866-552-2810 or [customerexperience@theubiquegroup.com](mailto:customerexperience@theubiquegroup.com). We're here to help!

## EVERY UBIQUE GROUP PRODUCT




Quality &  
Durability Tested



Packaged  
with Care



Shipped from  
our U.S. Facility

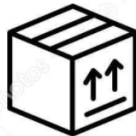

Designed  
for Easy  
Assembly





Ethically  
Sourced


Please visit <https://www.theubiquegroup.com/warranty> for complete warranty information.

**Read this manual before using this product. Failure to follow the instructions and safety precautions in this manual can result in serious injury or death. Keep this manual in a safe location for future reference.**


## PARTS:
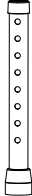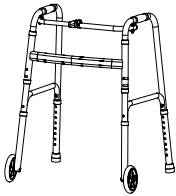
|   |                                                                                   |                    |   |                                                                                   |                          |
|---|-----------------------------------------------------------------------------------|--------------------|---|-----------------------------------------------------------------------------------|--------------------------|
| 1 |  | x2 Fixed Foot Legs | 2 |  | x1 Main Body with Wheels |
|---|-----------------------------------------------------------------------------------|--------------------|---|-----------------------------------------------------------------------------------|--------------------------|


### STEP1

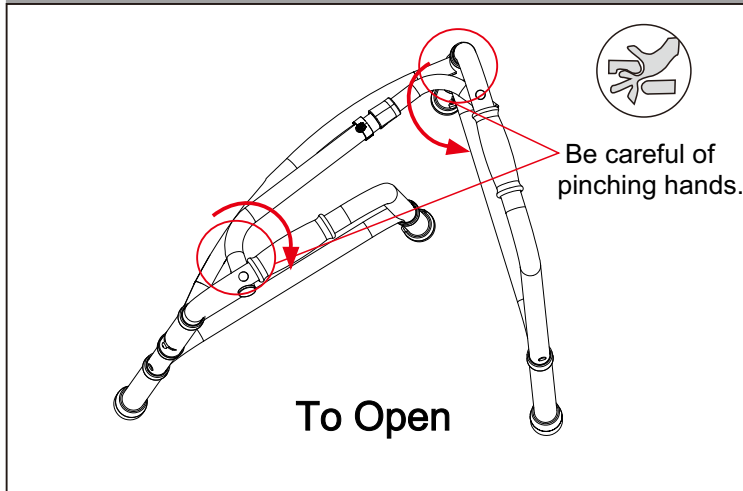


### STEP2

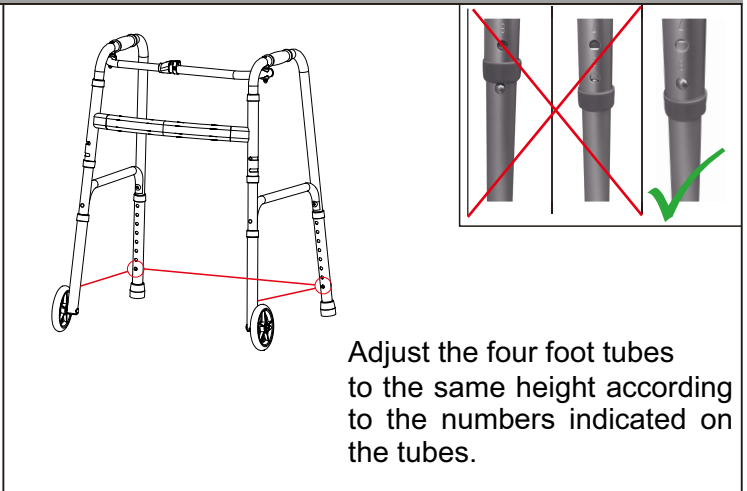


### STEP3

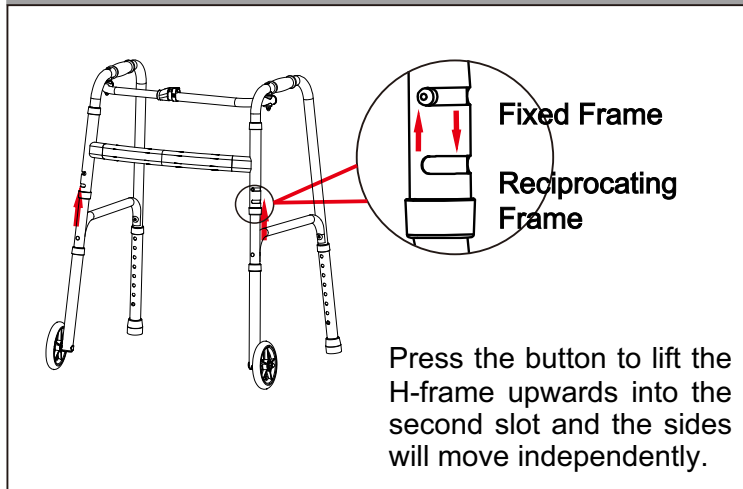


### STEP4

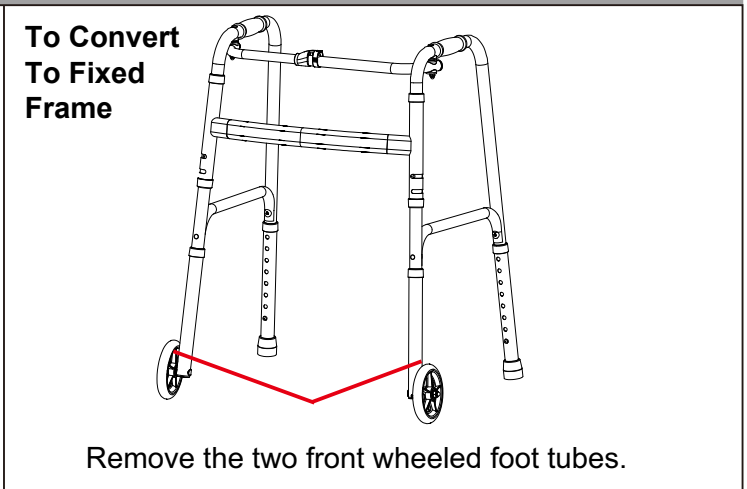

### STEP5

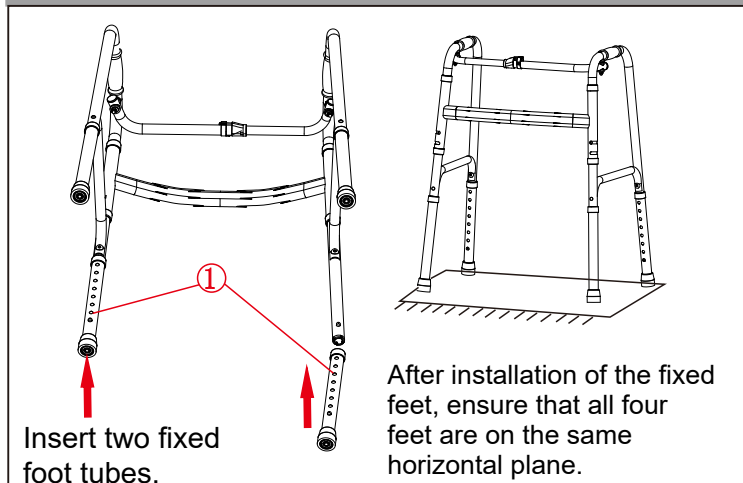

### STEP6

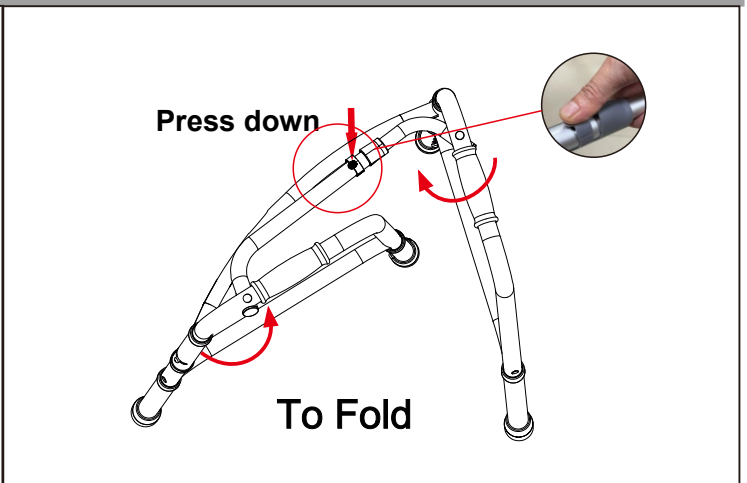

# Quality and Safety Guidelines

- Please read these safety instructions carefully before use and keep in a safe place for future reference.
- Consult your physician or supplier to determine the proper adjustment and use of this device.
- Regularly check all parts to ensure there is no damage to the tips, grips, or frame and that all screws are securely tightened.
- Hold firmly to the handles while using this product.
- Move forward with short steps to ensure proper balance.
- In case of falling, let the walker fall to the side to avoid contact with your body.
- Please follow the static and dynamic weight capacity limits provided on this item. If you are unsure of the weight capacities for this product, email us at [customerexperience@theubiquegroup.com](mailto:customerexperience@theubiquegroup.com) or call 866-552-2810.
- Before use, always ensure that the product is securely locked in the OPEN position, all attachments are locked in place, and all legs/feet are in contact with the flooring surface.
- DO NOT hang anything on the walker.
- This product is intended for adult use.
- While using this product, beware of any indentations, holes, obstacles, or other ground deformities in the path of the walker.
- This product is not intended for use as a lever or for moving furniture.
- DO NOT use on wet or icy surfaces or on stairs due to risk of injury.
- No modification may be made to this device. Use only original parts supplied by the manufacturer to replace any broken or defective parts.
- DO NOT store outdoors or in a vehicle and protect from extreme temperatures (Over 100°F or Below 32°F).
- Please inspect this product regularly to make sure there are no damaged or worn parts. Also inspect to make sure all hardware is tightened correctly and securely. For inspection instructions, please email us at [customerexperience@theubiquegroup.com](mailto:customerexperience@theubiquegroup.com) or call 866-552-2810.
- For maintenance instructions and replacement parts, please email us at [customerexperience@theubiquegroup.com](mailto:customerexperience@theubiquegroup.com) or call 866-552-2810.
- If any parts are damaged or worn, please stop using the product. Repair this product with manufacturer supplied parts only.
- Please visit [www.theubiquegroup.com](http://www.theubiquegroup.com) for our product warranty details or email us at [customerexperience@theubiquegroup.com](mailto:customerexperience@theubiquegroup.com) or call 866-552-2810.

## INTENDED USE:

Walking aids are used as part of a rehabilitation program when the user is recovering from an injury or surgery or as a long-term aid when the user has a permanent condition that causes difficulty with walking. This device helps maintain balance and aids in the prevention of injuries (particularly falls) allowing individuals to independently attend to their needs. Mobility aids may be used indoors or outdoors, depending on the model. Mobility aids are used to modify weight distribution so some of the weight carried through the legs while walking is transferred to the arms of the walker frame for support.

## Do Not Use If You Have Any Of The Following:

Severe Cognitive Disorder  
Severe Loss of Balance  
Severe Motor or Perception Disorders  
Weakness in the Upper Limbs

Please review and retain all assembly instructions, safety and quality guidelines and product warnings for future reference.